PLANO BASCULANTE INFANTIL

F.10N

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CHILDREN TILTING TABLE F.10N

For children from 3 to 13 years old(approx.).

The Layout of the board and its components allow for the work to be carried out whether in the prone or supine position.

Reclines using a telescopic system.

Removables feet rest to allow for respiratory exercises to be carried out. For caster with brake.

Removable head rest.

Supplied whith a trhoracic strap and two for the legs.

Metal structure, with chrome and epoxy finish.

Fireproof mataterial (M2)

Dimensions: From 115 to 150 cms. high, 60 cms. wide.

USER MANUAL

In the event of a program for reduced movement, strength and balance; these equipment's allow to gradually reach the static vertical position or supine when the patient is not able to carry his own weight on the lower limbs.

The vertical position is produced in a linear manner and with minimal effort on the part of the operator, with maximum comfort for the patient.


IMPORTANT WARNINGS!

When using a tilting table – standing frame the centre of gravity is always relatively high up. A person using a tilting table/standing frame is always dependent on the stability of the device, from the supporting points against the body and support area to the floor.

Our main objective was to develop a device, which satisfies the demands of many medical indications. Nevertheless, a tilting table/standing frame cannot be used without exception in all situations. The application and utilization should always be followed through with sense and reason.

- The tilting table / standing frame may only be operated with continuous presence of a qualified person, respectively a family member of the user.
- It may only be used by persons of a suitable stature and physical aptitude.
- Never place a patient in a standing aid, which is not coordinate with his or hers body weight and height.
- Do not place any hot containers or objects on the therapy table, as hereby the wood or coating could be seriously damaged.
- Do not place any heavy objects on the device (for example on the table) as it would affect the centre of gravity and consequently impair the balance.
- Do not operate or take any action if unsure of its effects; if in doubt please contact the nearest technical service centre or manufacturer.
- Always ensure there are no obstacles in operating range of the device, such as carpets, stairs or objects on the floor, which could lead to the device overturning.
- Never use the device on rough terrain, steeply sloping grounds or stairs.
- Never leave the user unattended with the device.
- Do not use without ensuring a correct mounting of the safety belts.
- Never place the device on steep grades.
- Do not use the device if any parts are damaged or missing.
- In case of malfunction or accident contact the reseller from whom you purchased the device from. Control before every use, that all adjustable parts are always locked.
- Check regularly if all screws and handles etc. are tightly fastened. Worn and defect parts are to be repaired or replaced immediately. This is specially important to ensure a safe use of the aid. Carefully check for a firm hold of the clamping jigs and the like.
- Adjust settings only in the unladed condition.
- Be careful when moving the device, whilst a patient is standing in the device. This is particularly applicable when crossing ramps, thresholds and similar. Never allow person not familiar to operate the standing aid respectively those who have not read this operating manual.

1. SWING SYSTEM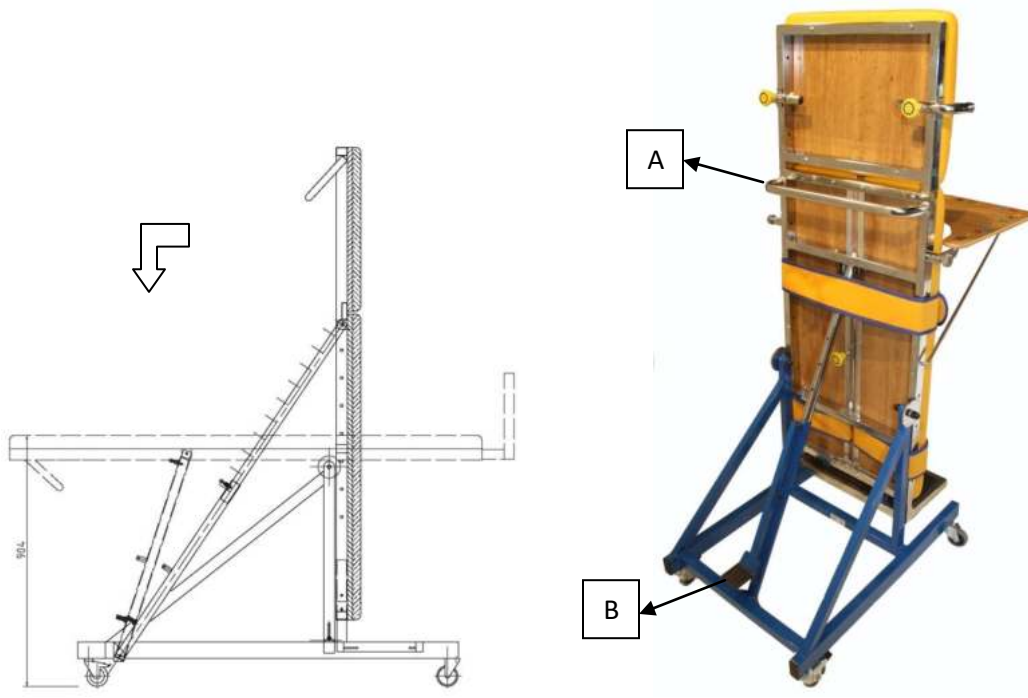

To swing the tilted table to the horizontal position, step on the pedal B and while assisting the movement pulling from the handle A towards the indicated arrow direction.


2. ACCESORIES:

INCLUDED ACCESORIES:

- (1) 1 THORACIC STRAP
- (2) 2 LEGS STRAP
- (3) REMOVABLE HEAD REST
- (4) REMOVABLE FEET REST

OPTIONAL ACCESORIES:

- (5) F.10N-A - TRAY
- (6) F.10N-B - SET OF HEAD SUPPORTS
- (7) F.10N-C - SET OF CHEST SUPPORTS
- (8) F.10N-D - LEGS SEPARATOR
- (9) F.10N-E - ABDUCTION ROLL
- (10) F.10N-F - CHILDREN FEET REST (for younger children)

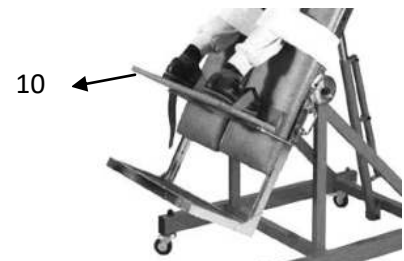3. POSITIONING ACCESSORIES

- REMOVABLE HEAD REST
- REMOVABLE FEET REST
- F.10N-A - TRAY
- F.10N-B - SET OF HEAD SUPPORTS
- F.10N-C - SET OF CHEST SUPPORTS
- F.10N-D - LEGS SEPARATOR
- F.10N-E - ABDUCTION ROLL
- F.10N-F - CHILDREN FEET REST (for younger children)

4. WARRANTY:

ORTOTEC莎, S.L. will provide a warranty for two years from the date of purchase, the working of this product against manufacturing defects. Our warranty includes repair, replacement, or exchange of the product and / or components including labor in our installations.

To have effect this guarantee, is required the purchase invoice.

THIS WARRANTY SHALL NOT UNDER THE FOLLOWING CONDITIONS:

When the use, care and operation of the product has not been done in accordance with the instructions contained in the operating instructions.

When the product has been used outside of its capacity, abused, beaten, exposed to moisture, ingressed by any liquid or corrosive substance, as well as any other fault attributable to the consumer.

When the product has been disassembled, modified or repaired by unauthorized persons.

The breakage and / or replacement of the bulb are not part of the warranty for this product.

In our continuous improvement process, Ortotec莎 reserves the right to modify any characteristic without prior notice.