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MOBILE TILT TABLE - F.12

ELECTRIC - F.12/2

The Table tilts from the vertical position to the horizontal position.

The multiple tilting positions are achieved by a pedal that transmits and fixes the chosen position by its telescopic component. (F.12)

The multiple tilting positions are achieved by an electric actuator, acting as a push-button control or pedal (to be chosen). (F.12 / 2)

The Table has a removable footrest. Its coupling is carried out by telescopic system, blocked by two synchronized lateral pins.

The frame, in its upper and rear part, has a transverse handle that facilitates the movement of tilting and transport.

It has four wheels to facilitate the movement, the two later with brakes to achieve immobility.

It is supplied with a chest strap and two for lower extremities. Metallic structure finished in chrome and polyester.

Fire retardant upholstery.

Measures: 175 x 74 cm.

USER MANUAL

Manufacturer: Ortotecsa,s.l. C/Peña de Francia,8 – 28500 Arganda del Rey (Madrid), España.

Tlf.: +34918719132 / Fax: +34918719323 – email: info@ortotecsa.com, www.ortotecsa.com

In the context of a program of reeducation of movement, strength and balance these equipments allow to gradually reach the vertical position when the patient is not able to carry his own weight on the lower limbs.

The vertical position is produced in a linear manner and with minimal effort on the part of the operator, with maximum comfort for the patient.

IMPORTANT WARNINGS!


When using a tilting table – standing frame the centre of gravity is always relatively high up. A person using a tilting table/standing frame is always dependent on the stability of the device, from the supporting points against the body and support area to the floor.

Our main objective was to develop a device, which satisfies the demands of many medical indications. Nevertheless, a tilting table/standing frame cannot be used without exception in all situations. The application and utilization should always be followed through with sense and reason.


- The tilting table / standing frame may only be operated with continuous presence of a qualified person, respectively a family member of the user.
- It may only be used by persons of a suitable stature and physical aptitude.
- Never place a patient in a standing aid, which is not coordinate with his or hers body weight and height.
- Do not place any hot containers or objects on the therapy table, as hereby the wood or coating could be seriously damaged.
- Do not place any heavy objects on the device (for example on the table) as it would affect the centre of gravity and consequently impair the balance.
- Do not operate or take any action if unsure of its effects; if in doubt please contact the nearest technical service centre or manufacturer.
- Always ensure there are no obstacles in operating range of the device, such as carpets, stairs or objects on the floor, which could lead to the device overturning.
- Never use the device on rough terrain, steeply sloping grounds or stairs.
- Never leave the user unattended with the device.
- Do not use without ensuring a correct mounting of the safety belts.
- Never place the device on steep grades.
- Do not use the device if any parts are damaged or missing.
- In case of malfunction or accident contact the reseller from whom you purchased the device from. Control before every use, that all adjustable parts are always locked.
- Check regularly if all screws and handles etc. are tightly fastened. Worn and defect parts are to be repaired or replaced immediately. This is specially important to ensure a safe use of the aid. Carefully check for a firm hold of the clamping jigs and the like.
- Adjust settings only in the unladed condition.
- Be careful when moving the device, whilst a patient is standing in the device. This is particularly applicable when crossing ramps, thresholds and similar. Never allow person not familiar to operate the standing aid respectively those who have not read this operating manual.

1. TILTING SYSTEM



- MODEL F.12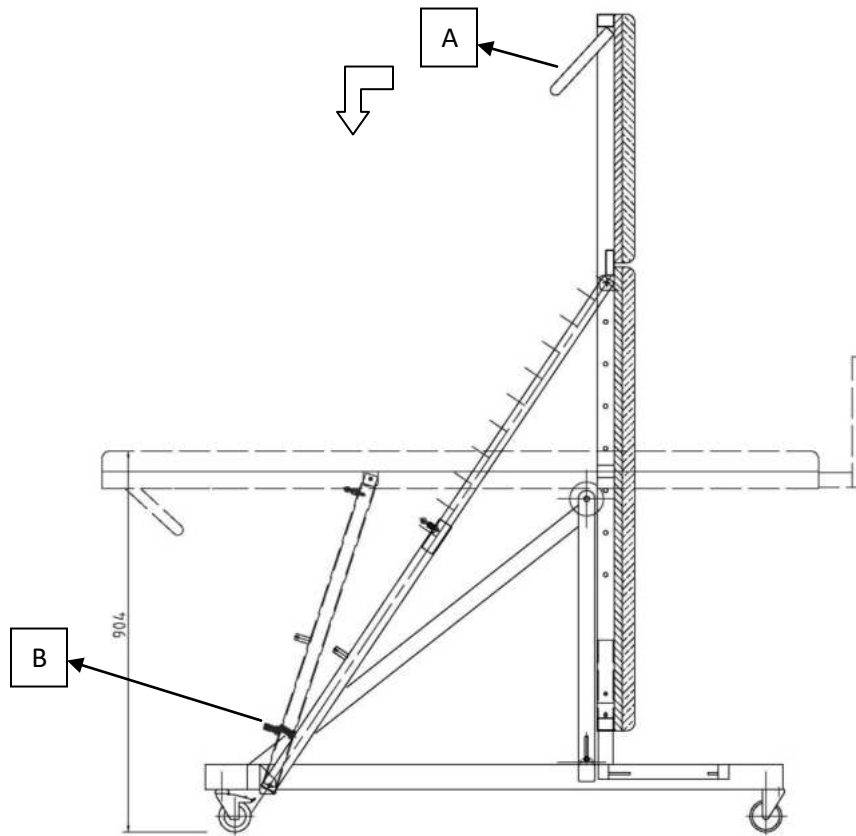

To tilt the table to the horizontal position, step on the pedal B and while assisting the movement pulling from the handle A towards the indicated arrow direction.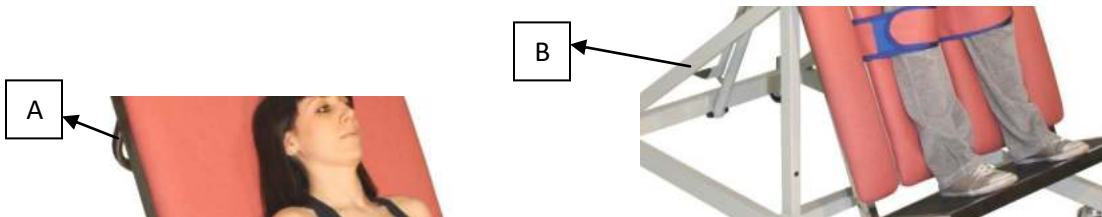

- **MODEL F.12/2 ELECTRIC**

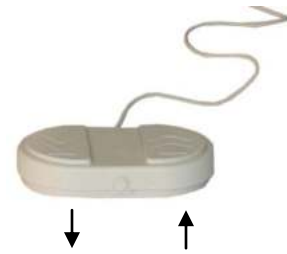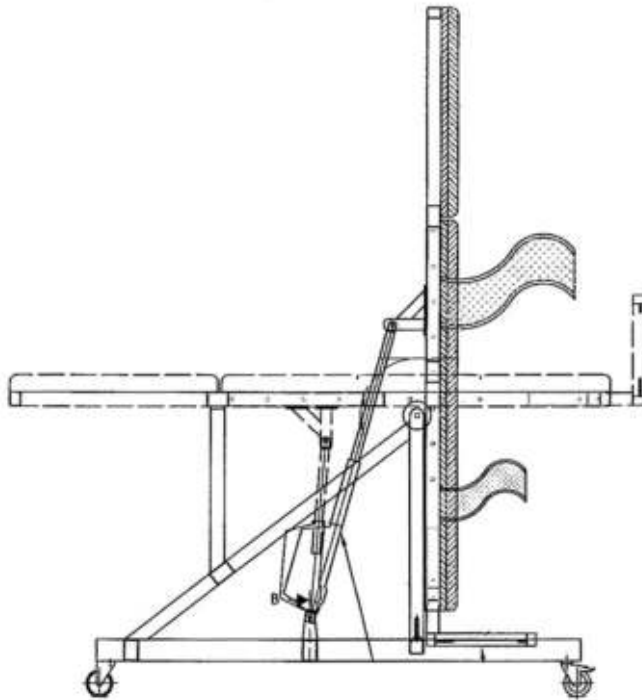For tilting the table to the horizontal position, step on the electric pedal in the desired direction.

2. ACCESORIES INCLUDED

- 1 THORATHIC STRAP
- 2 LOWER EXTREMITIES STRAP
- DETACHABLE FEET REST

3. OPTIONAL ACCESORIES

- ABDUCTION ROLL – F.12-C


- CUP FOR STABILISING FEET - F.12-D

- TRAY - F.12-E

- SET OF ARM RESTS - F.12-H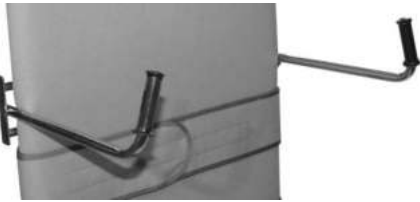

- SET OF HEAD SUPPORTS – F.12-I

- SET OF CHEST SUPPORTS – F.12-J

- LEGS SEPARATOR – F.12-L

4. TECHNICAL CHARACTERISTICS

Motor: 220-240V / 50-60Hz

110-127V / 50-60Hz 110V model

Maximum load: 180 Kg.

Tilt angle: Tilt 90° from vertical to horizontal position.

Power: 6000N (push)

4000N (shot)

Speed: 2.9 mm x second when pushing the maximum load of 6000N

4.2 mm x second without load.

5. WARRANTY:

ORTOTEC莎, S.L. will provide a warranty for two years from the date of purchase, the working of this product against manufacturing defects. Our warranty includes repair, replacement, or exchange of the product and / or components including labor in our installations.

To have effect this guarantee, is required the purchase invoice.

THIS WARRANTY SHALL NOT UNDER THE FOLLOWING CONDITIONS:

When the use, care and operation of the product has not been done in accordance with the instructions contained in the operating instructions.

When the product has been used outside of its capacity, abused, beaten, exposed to moisture, ingressed by any liquid or corrosive substance, as well as any other fault attributable to the consumer.

When the product has been disassembled, modified or repaired by unauthorized persons.

The breakage and / or replacement of the bulb are not part of the warranty for this product.

In our continuous improvement process, Ortotecsa reserves the right to modify any characteristic without prior notice.