PLANO BASCULANTE GRADUABLE

F.12/4

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ADJUSTABLE TILT TABLE – F.12/4

Electrical table, that can be tilted from a horizontal to a vertical position.

Height-adjustable from 45 to 95 cm.

Equipped with a dual action pedal with a non-slip base.

Two motors with built in safety lock. Easily detachable feet rest, which allows for patient to be put in the

prone/supine position to carry out breathing exercises.

The Electrical table provided with four casters with brake. And weigh 180Kg

Gas cylinder reclining head rest with breathing hole.

Supplied with a set of three straps for stabilising the thorax and lower extremities.

Dimensions: 195 x 74 x 45-95 cms, covered with 5 cm fix matters

The Capacity is 200kg

USER MAUNAL

MANUFACTURER: Ortotecsa,s.l. C/Peña de Francia,8 – 28500 Arganda del Rey (Madrid), España.
Tlf.: +34918719132 / Fax: +34918719323 – email: info@ortotecsa.com, www.ortotecsa.com

In the event of a program for reduced movement, strength and balance; these equipment's allow to gradually reach the static vertical position or supine when the patient is not able to carry his own weight on the lower limbs.

The vertical position is produced in a linear manner and with minimal effort on the part of the operator, with maximum comfort for the patient.


IMPORTANT WARNINGS!

When using a tilting table – standing frame the centre of gravity is always relatively high up. A person using a tilting table/standing frame is always dependent on the stability of the device, from the supporting points against the body and support area to the floor.

Our main objective was to develop a device, which satisfies the demands of many medical indications. Nevertheless, a tilting table/standing frame cannot be used without exception in all situations. The application and utilization should always be followed through with sense and reason.

- The tilting table / standing frame may only be operated with continuous presence of a qualified person, respectively a family member of the user.
- It may only be used by persons of a suitable stature and physical aptitude.
- Never place a patient in a standing aid, which is not coordinate with his or her body weight and height.
- Do not place any hot containers or objects on the therapy table, as hereby the wood or coating could be seriously damaged.
- Do not place any heavy objects on the device (for example on the table) as it would affect the centre of gravity and consequently impair the balance.
- Do not operate or take any action if unsure of its effects; if in doubt please contact the nearest technical service centre or manufacturer.
- Always ensure there are no obstacles in operating range of the device, such as carpets, stairs or objects on the floor, which could lead to the device overturning.
- Never use the device on rough terrain, steeply sloping grounds or stairs.
- Never leave the user unattended with the device.
- Do not use without ensuring a correct mounting of the safety belts.
- Never place the device on steep grades.
- Do not use the device if any parts are damaged or missing.
- In case of malfunction or accident contact the reseller from whom you purchased the device from. Control before every use, that all adjustable parts are always locked.
- Check regularly if all screws and handles etc. are tightly fastened. Worn and defect parts are to be repaired or replaced immediately. This is specially important to ensure a safe use of the aid. Carefully check for a firm hold of the clamping jigs and the like.
- Adjust settings only in the unloaded condition.
- Be careful when moving the device, whilst a patient is standing in the device. This is particularly applicable when crossing ramps, thresholds and similar. Never allow person not familiar to operate the standing aid respectively those who have not read this operating manual.

1. MEASURES: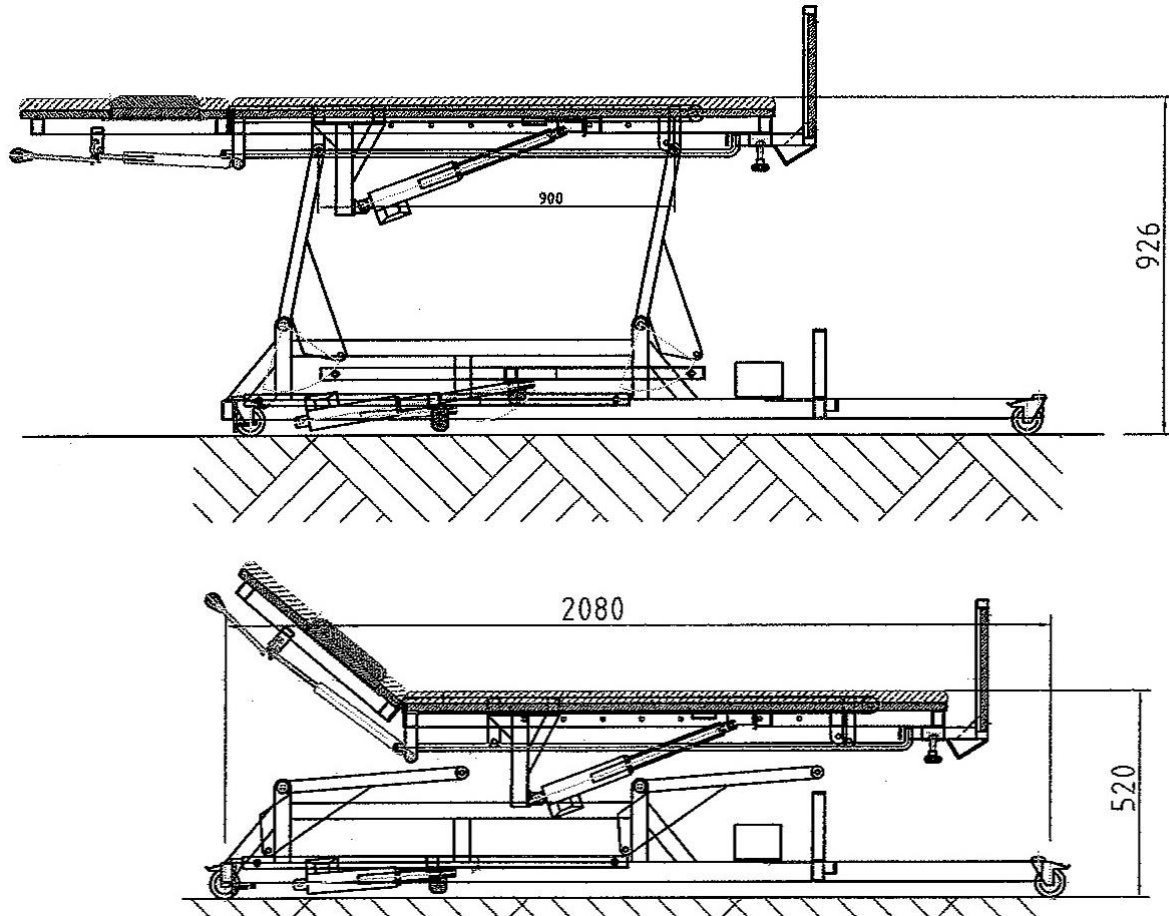


2. Using Mode:

Headboard: To raise and lower, use the lever "A" and help the movement with the hand to the desired height.

Facial ventilation: Pull Up the strap "B" to remove the cover.

Belts: Turn over the bars "C" provided under the plane and join the center of the belt ends "D".

Footrests: To change the position, loosen the two knobs "E", move the footrest until you find the appropriate locking position and firmly tighten the knobs.

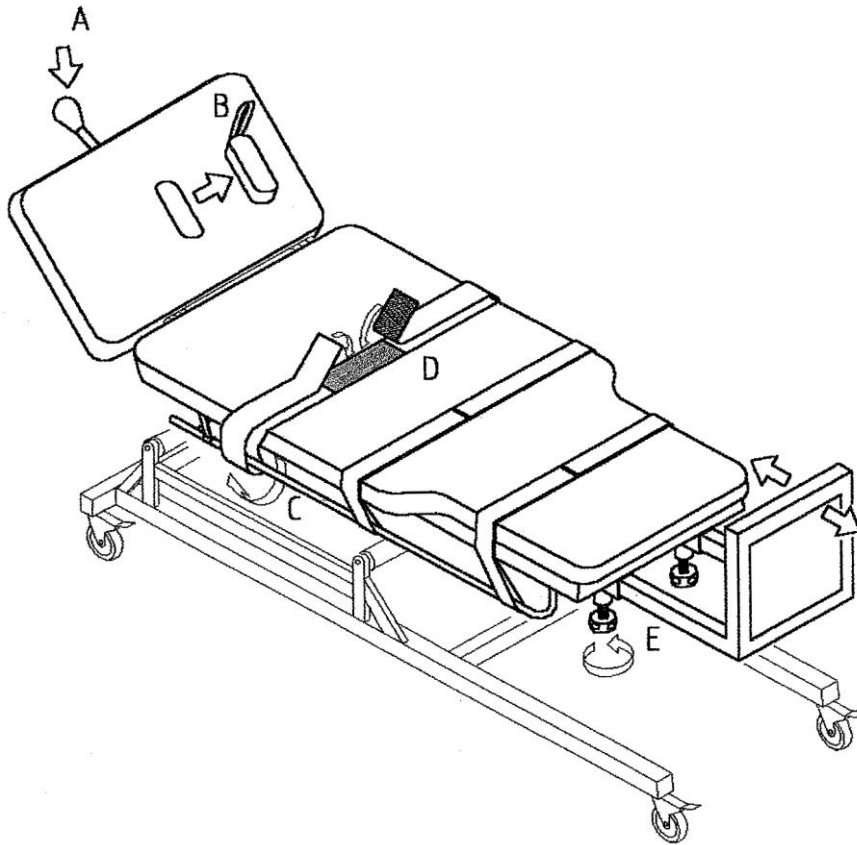
OPTIONAL ACCESORIES:

- TRAY - F.12-E
- ARMREST - F.12-H



3. BACULING AND ELEVATION: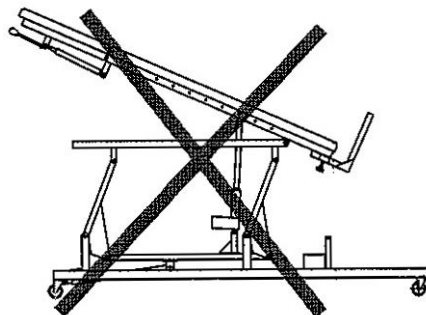

ATTENTION!
DO NOT TILTE THE PLANE
BEING ELEVATED
AND DO NOT RAISE THE
PLANE BEING TILTED


Very Important:

- Before Tilting, make sure that the footrest is in one of the two locking positions provided and that the knobs are firmly tightened.
 - Do not activate the tilting if the plane is not at the minimum height of elevation
 - Do not activate the elevation if the plane is not fully horizontal.
- Improper use could cause damage to the mechanical system of the device.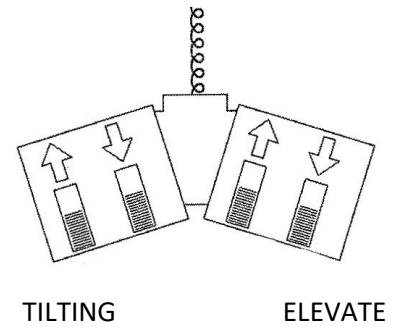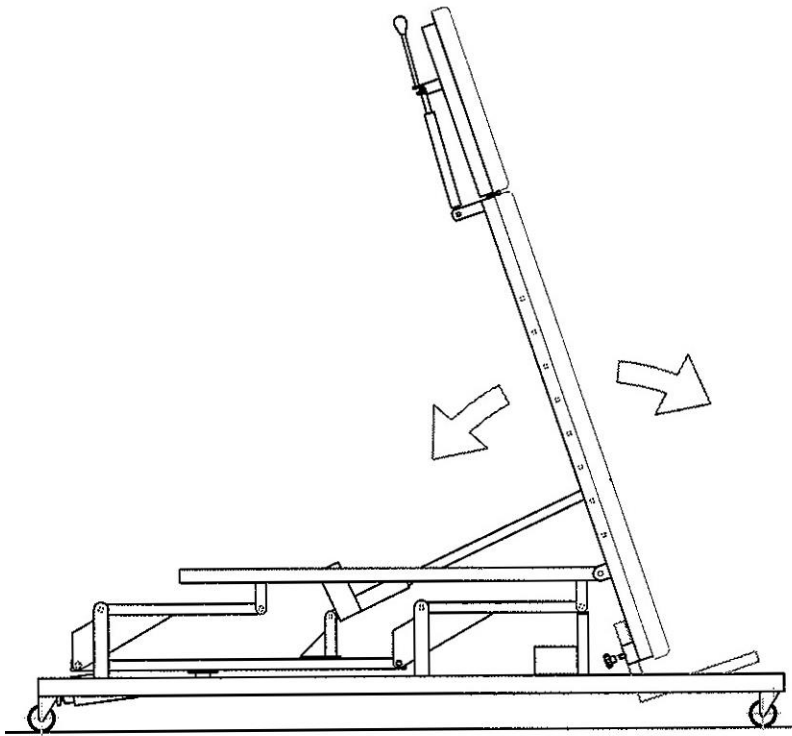

TILTING

ELEVATE

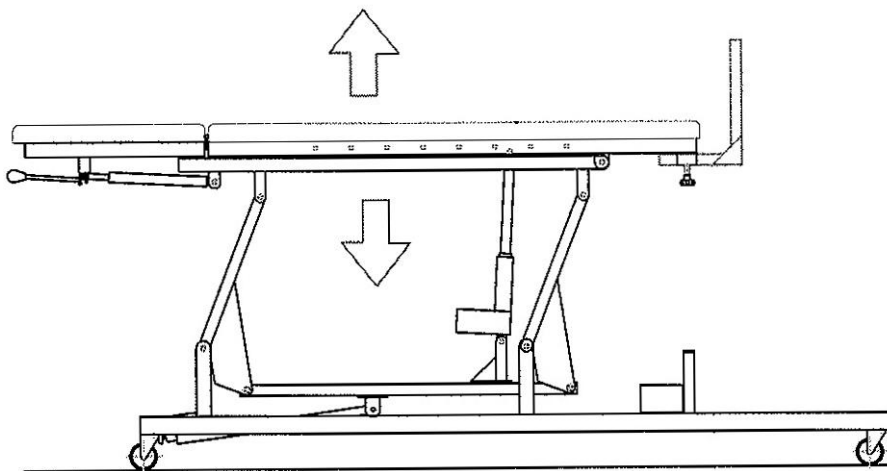
UP

Down

4. Electrica Installation: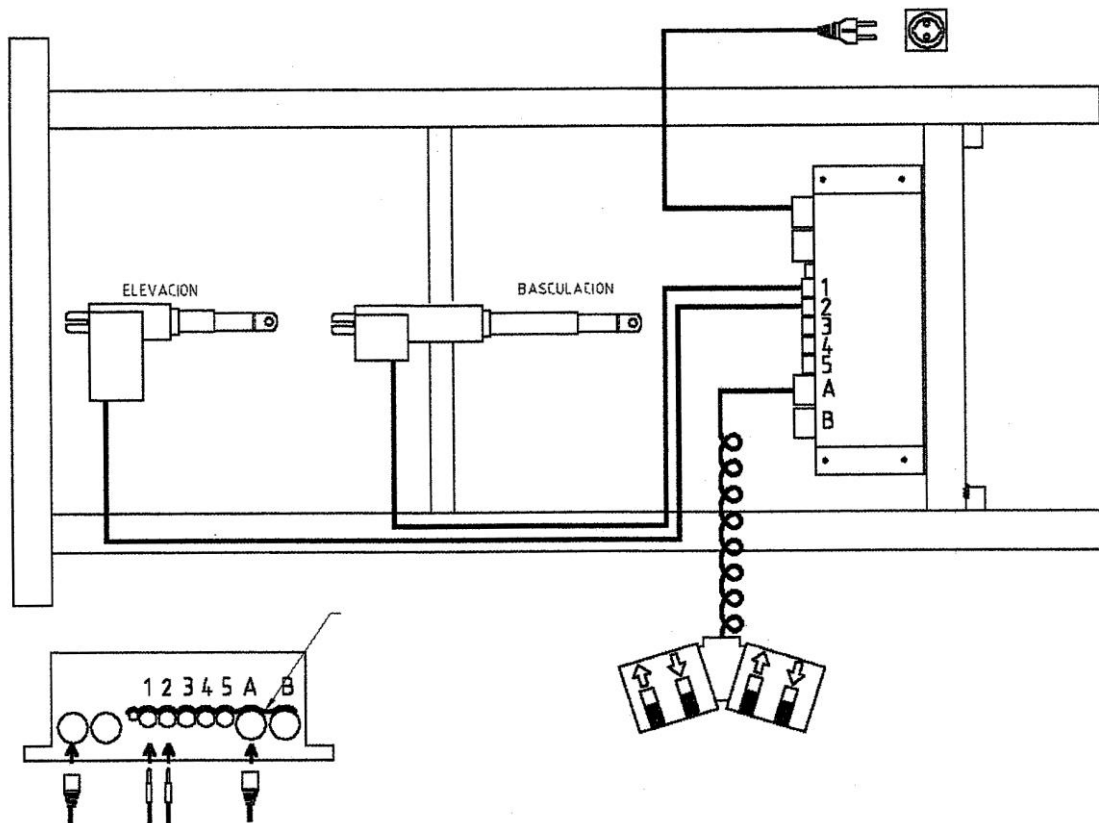

5. WARRANTY:

ORTOTECESA, S.L. will provide a warranty for two years from the date of purchase, the working of this product against manufacturing defects. Our warranty includes repair, replacement, or exchange of the product and / or components including labor in our installations.

To have effect this guarantee, is required the purchase invoice.

THIS WARRANTY SHALL NOT UNDER THE FOLLOWING CONDITIONS:

When the use, care and operation of the product has not been done in accordance with the instructions contained in the operating instructions.

When the product has been used outside of its capacity, abused, beaten, exposed to moisture, ingressed by any liquid or corrosive substance, as well as any other fault attributable to the consumer.

When the product has been disassembled, modified or repaired by unauthorized persons. The breakage and / or replacement of the bulb are not part of the warranty for this product.

In our continuous improvement process, Ortotecsa reserves the right to modify any characteristic without prior notice.