SISTEMA DE POLEAS / PULLEY SYSTEM

INDIVIDUAL - F.20/1

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SINGLE – F.20/1

Dual function pulley system to be used for both upper and lower limbs.

The complete set of pulleys is placed on a varnished wooden board.

Pulleys are put together with double set of steel ball bearings, in order to keep the machine as silent as possible.

Pulley fitted with following grooved weights: 3 x 2 kg., 1 x 1 kg., and 1 x 1/2 kg.

TWIN– F.20/2

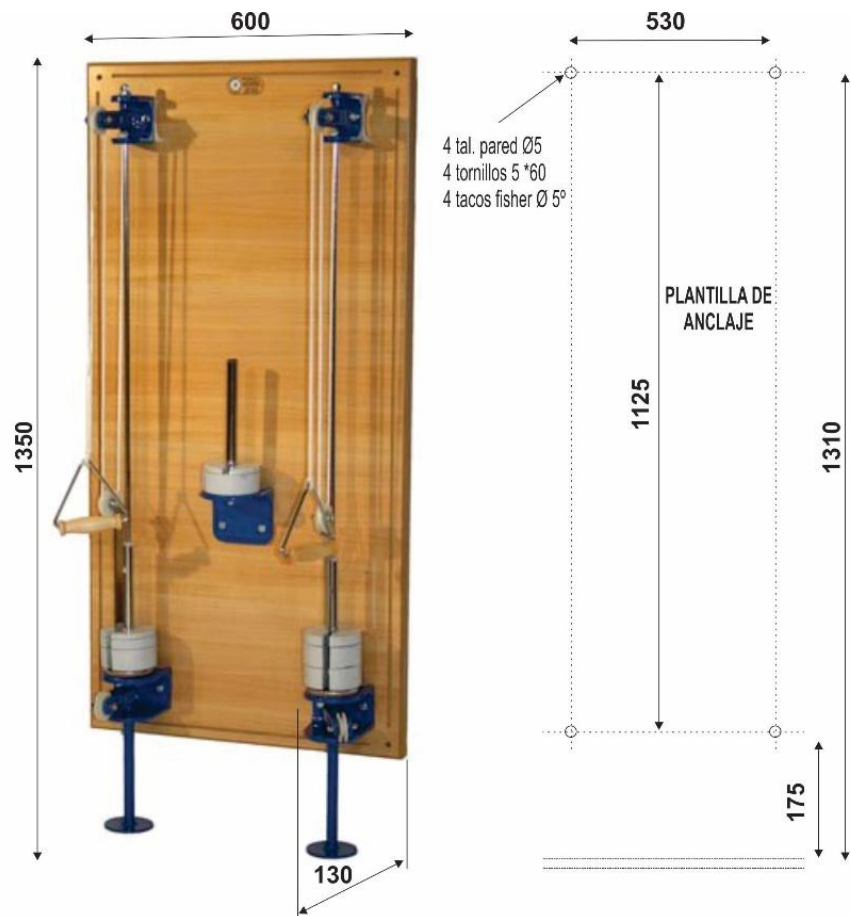
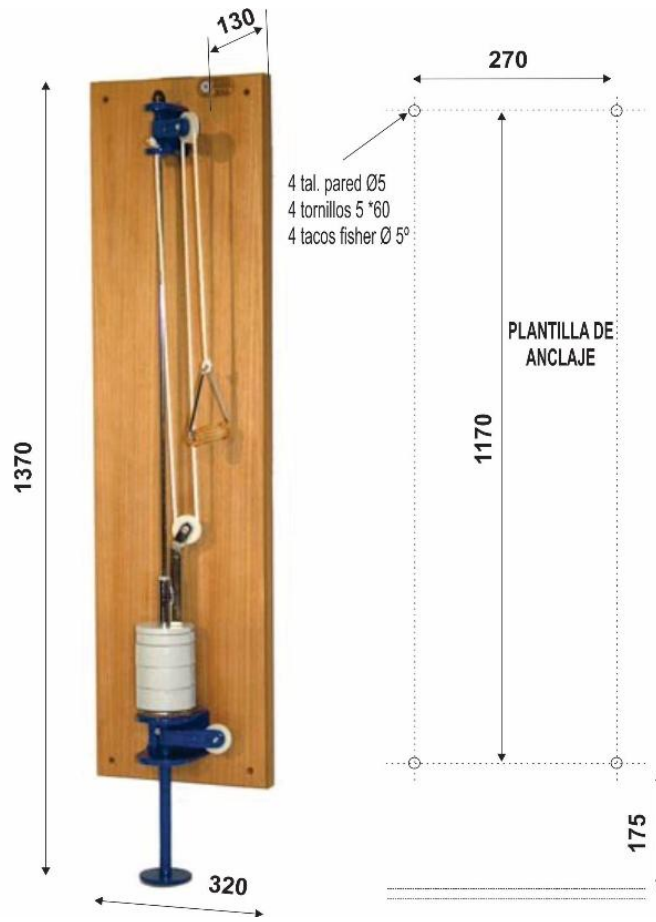
Adjustable dual function bilateral pulleys to be used for both upper and lower limbs.

The complete set of pulleys is placed on a varnished wooden board, with a base for the weights to be

used placed at the centre of the set.

Double set of steel ball bearings.

Pulley fitted with following grooved weights: 4 x 2 kg., 2 x 1 kg., and 2 x 1/2 kg.

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RECOMMENDATIONS FOR USE:

With these pulleys is possible to perform active or resistance movements. We can use the pulleys in different ways, with or without resistance. We have to place the weights, whenever we possibly can, within the visual field of the patient.

As a gripping point we will use the handles, which help to traction and grip. In those cases where the patient does not have enough pressure force with the hand that grabs the handle, it can be used in that case the holding mit G.17003 (optional).

To exercise the lower limbs, it will be necessary to use foot stirrups G.17002 (optional).

It is a treatment with a **few contraindications**. It is not suitable for people with recent bone fractures, ankylosis or when the patient does not meet the physical or mental capacity to perform this treatment.

APARATOS Y DISPOSITIVOS
PARA APLICACIONES TERAPEUTICAS
ORTOTECSA
ORTOTECSA, S.L.
C/.Peña de Francia,8 (28500) Arganda del Rey
MADRID (España)
Tlf: +34 918 719 132 / Fax: +34 918 719 323
E-mail: info@ortotecs.com
Web: www.ortotecs.com

Fabricante / Manufacturer: Ortotecsa,s.l. C/Peña de Francia,8 – 28500 Arganda del Rey (Madrid), España.
Tlf.: +34918719132 / Fax: +34918719323 – email: info@ortotecs.com, www.ortotecs.com