

SOPORTE TRACCIONES

SOPORTE TRACCIONES - F.25

Para fijación en pared.
De construcción metálica, rematado en poliéster.
Incluye: mosquetón, cuerda, percha y fronda.

SOPORTE TRACCIONES GRADUABLE - F.25/0

Para fijación a cama.
Provisto de tres poleas.
Graduable en altura.
De construcción metálica, rematado en cromo y poliéster.
Incluye: 3 m. de cuerda y 2 mosquetones.

SOPORTE TRACCIONES ARTICULADO - F.25/2

Para fijación en puerta o espaldera.
Construido en aluminio, de fácil plegado.
Incluye: mosquetón, cuerda, percha y fronda.

TRACTION SUPPORT

TRACTION SUPPORT – F.25


For fixing to the wall.
Made from metal with epoxy finish.
Includes: snap hooks, cord, hanger and hand mitt.

ADJUSTABLE TRACTION SUPPORT – F.25/0

To be fixed to a bed.
Equipped with three pulleys.
Made from metal with chrome and epoxy finish.
Includes: 3 mts. cord and two snap hooks.

ARTICULATED TRACTION SUPPORT – F.25/2

To be fixed to a door or wall ladder.
Made from aluminium, easily foldable.
Includes: snap hook, cord, hanger and hand mitt.

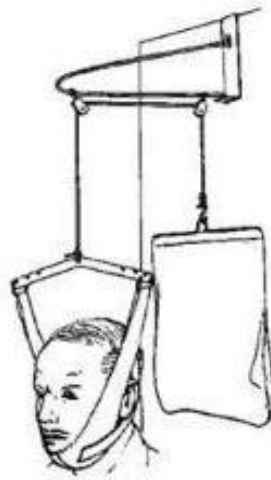



RECOMENDACIONES DE USO

La Tracción cervical se puede aplicar de forma continua o intermitente. En el caso de la tracción continua el peso es menor que en la intermitente. En ambos casos se colocará el barbuquejo con apoyo mentonario, siendo el peso recomendado de 5 a 15 Kg.

Siga cuidadosamente todas las instrucciones dadas por su médico, fisio o terapeuta para el uso del producto.



RECOMMENDATIONS OF USE

The vertical traction can be applied for continuous or intermittent way. In case of continuous traction the weight is less than in the intermittent traction. In both cases, the chin strap will be placed with supported chin, being the recommended weight between 5 and 15 kg.

Follow carefully all instructions given by your doctor, physic or therapist for the use of the product.



RECOMENDACIONES DE INSTALACIÓN / RECOMMENDATIONS OF INSTALLATION (F.25)

