# CAMILLAS F.28

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1

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# TREATMENT TABLES F.28 USER MANUAL

9

We offer you an ideal table for every situation. The different lines in our range are designed to meet every need. In any table that you choose will be guaranteed the stability, quality and reability. All our models and accessories are made from materials of the highest quality. They are design for long-term intensive use.

The facial opening provides two great benefits.

First, for the treatment of the area neck/shoulder, the patient may be positioned very close to the cranial border. This means that the area is easily accessible for treatment without having to bend over.

Secondly, each patient sits comfortably in the most suitable position regardless of its height.

Both high and low people can therefore be positioned in flexion around the thoracic-lumbar region while the head is properly supported in the hole.

In the 3-sections models the flexion position and/or Trendlenburg position (inclination of the whole body) is possible.

The 5-sections models offer all the features and advantages of the 3-sections models with an extra: an armrest adjustable separately. These features allow the patient to assume more relaxed position, which improves the treatment effect.

The Bobath tables are characterized as exercise therapy for neurological movement. These models have been specially developed for use with the methods of Bobath and Vojta exercise.

This large area offers you and your patient ample space.

## TYPES OF STANDARD TABLES:

Headrest adjustable lateral control, provided with facial breathing hole.

Mobile base with double brake lever.

Graduation height by electric activator (electric tables).

Hydraulic system activator pedal on both sides (hydraulic tables).

Metal frame finished in chrome and epoxy.

Fireproof upholstery (global skay “blue jeans”)

Maximum recommended weight: 130 Kg. (hydraulic tables)

Maximum recommended weight: 180 Kg. (electric tables)

Measures: 195 x 70 x 50-100 cm (2-sections table)

Measures: 200 x 100 x 50-100 cm (Bobath table)

10

### **TWO SECTIONS TABLE ELECTRIC – F.28-E2 / HYDRAULIC – F.28-H2**

### **BOBATH TYPE TABLE ELECTRIC - F.28-EB / HYDRAULIC – F.28-HB**

By its design and measures, is especially suitable for treatment according to the “Bobath” techniques.

Extra large headrest for support functions if required.

## TYPES OF TABLES DELUXE:

Headrest adjustable in two directions (up and down) by gas cylinder.

Headrest provided with facial breathing hole.

Mobile base with lifting mechanism of the wheels centralized, operating by a single lever.

Graduation height by electric activator (electric tables).

Hydraulic system activator pedal on both sides (hydraulic tables)

Metal frame finished in chrome and epoxy.

Fireproof upholstery (SKAY HIGHLANDS APRICOT 203-2025)

Maximum recommended weight: 130 Kg. (hydraulic tables)

Maximum recommended weight: 180 Kg. (electric tables)

Measures: 195 x 70 x 50-100 cm (2,3 and 5-sections table)

Measures: 200 x 100 x 50-100 cm (Bobath table)

**TWO SECTIONS TABLE DELUXE ELECTRIC – F.28-8E2 / HYDRAULIC – F.28-8H2**

**THREESECTIONS TABLE DELUXE ELECTRIC – F.28-8E3 / HYDRAULIC – F.28-8H3**

Headrest, central plane and footboard adjustable by gas cylinder, placed on both sides.

**FIVE SECTIONS TABLE DELUXE ELECTRIC – F.28-8E5 / HYDRAULIC – F.28-8H5**

Five adjustable sections: head, two independent for the arms, body and legs.

Boards for the head, body and legs all can be adjusted using gas cylinder, a control for which is located on either side.

**BOBATH TYPE TABLE DELUXE ELECTRIC - F.28-8EB / HYDRAULIC – F.28-8HB**

Due to its design and dimensions, is especially recommended for treatments using "Bobath" techniques.

Extra large headrest for support functions if required.

## INSTRUCTIONS

### **Security instructions**

To raise or lower the table, make sure that neither the patient nor the therapist is trapped between the moving parts of the hoist with the hands, feet or other body parts.

### **Cleaning**

The table can be cleaned with a damp cloth, using a liquid household cleaner if desired.  
For disinfecting, alcohol solution at 70% can be used.


### **Operating instructions**

Managing the treatment table is easy and secure. Illustrations show you how to adjust the various positioning:

12

#### **7. Height adjustment:**

- Hydraulic tables: To raise the stretcher, step on the pedal “S” repeatedly until the desired height. To lower, step on the pedal “B”.

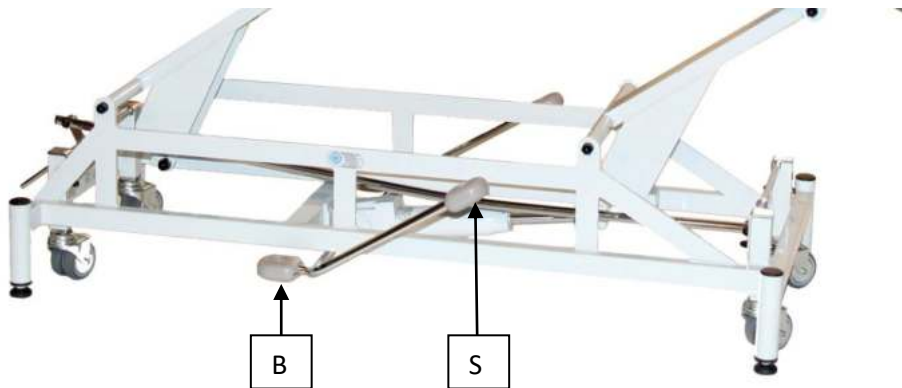- Electric tables: To raise or lower, step on the foot control in the direction of the arrows  $\uparrow\downarrow$  until the desired height.


**8. Shifting and braking:**

To move the table, down the lever with your foot until it is as shown in the picture.  
To restrain the table and leave it in the desired position, release the wheels rising the lever up.

**9. Facial breathing hole:**

Pull the hook tape to remove the lid.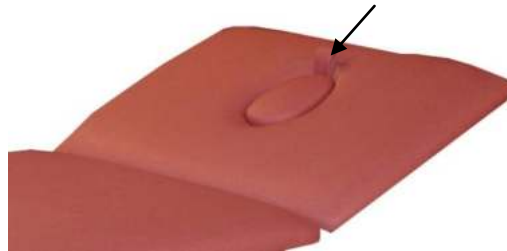

**10. Headrest or footboard adjustment:**

To raise or lower the headrest (the headrest only lower in the table “Deluxe” models) acting on the lever “A” and helping the movement with your hand to the desired height.


**; ATTENTION! ONLY FOR THE 3 AND 5 SECTION TABLES:**

**11. Adjustment of the center section to flexion position:**

To put the plane on flexion position (section 2) act on one of the levers “F” this may be found on both sides of the table.


The lever “P” serves to raise the section 3.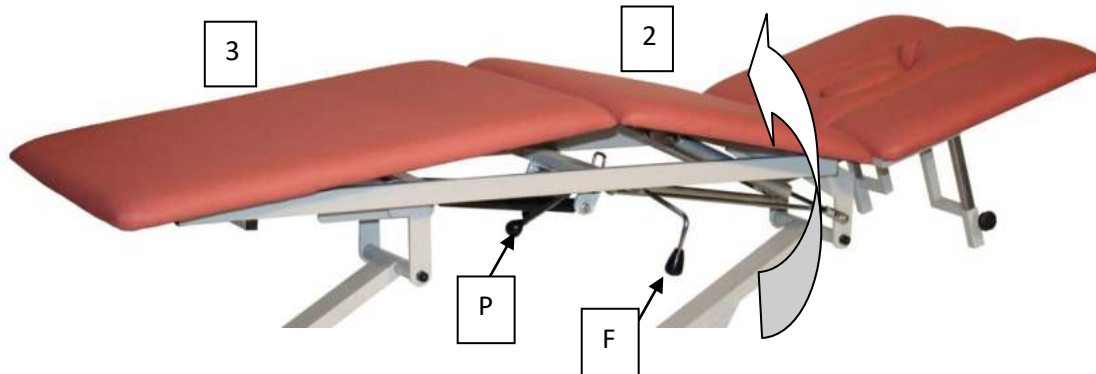

**12. Adjustment of the armrest:**

To raise and lower the headrest, act on the lever “A” as shown in the point 4. The armrests will swing simultaneously in the same direction as the head.

Once adjusted, you can adjust the height of each armrest loosening the knobs (X) and turning them tighter at the desired height.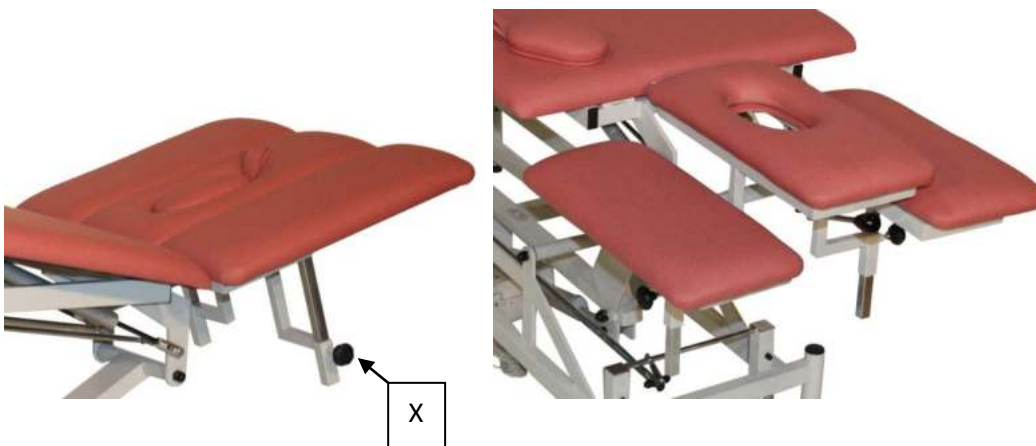

**Warranty**

ORTOTEC莎, S.L. will provide a two year warranty against manufacturing defects from the date of purchase. Our warranty covers the repair, replacement or exchange of the product and / or components, including labor performed at our facilities.


To take effect this warranty, the bill of sale is required.

This warranty does not apply under the following conditions:

- When the use and operation of the product have not been conducted in accordance with the given instructions.
- When the product has been used beyond its capacity, abused, beaten, exposed to moisture, attacked by any liquid substance or corrosive, as well as any failure attributable to the consumer.
- When the product has been disassembled, modified or repaired by unauthorized persons.

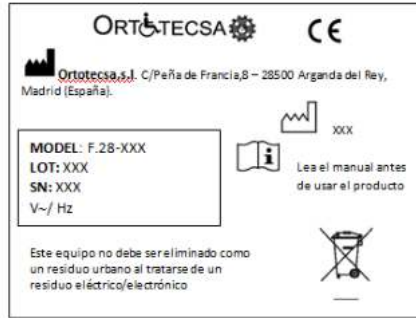## LABEL

	MANUFACTURER
	MANUFACTURE DATE (YEAR/MONTH)
<b>SN</b>	SERIAL NUMBER
<b>LOT:</b>	LOT NUMBER
<b>MODEL:</b>	PART NUMBER OF DEVICE
	CONSULT THE ACCOMPANIMENT DOCUMENTATION
  $V_{ac}$	AC (ALTERNATE CURRENT)
	The device must not be disposed as an urban waste, because it is an Electrical and electronic waste

**DIFERENT POSSITIONS**