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F.42/1

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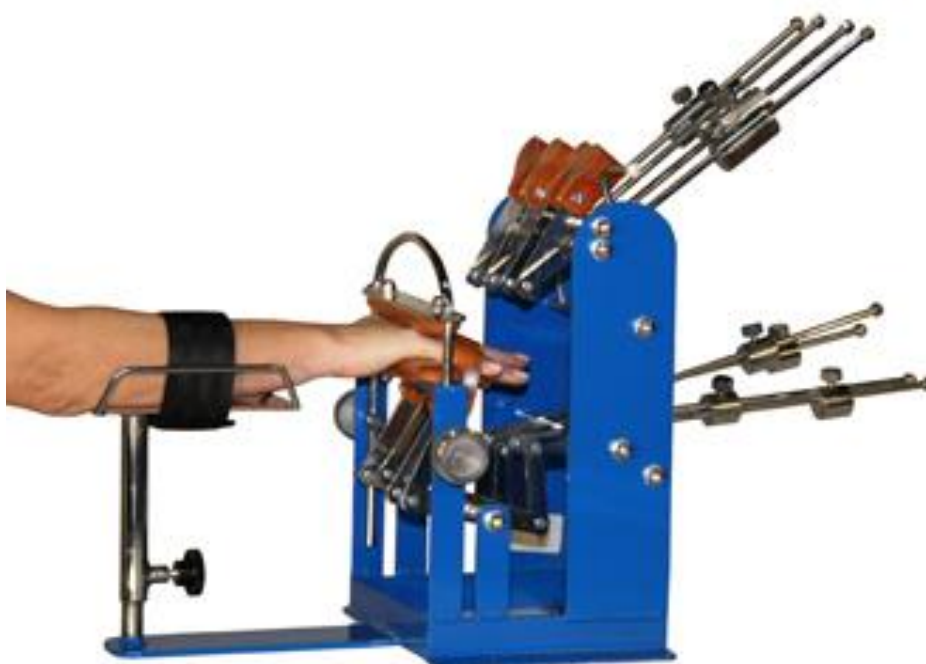
FINGER EXERCISE MACHINE - F.42/1

Comprises:

Adjustable support to hold forearm in position.

Padded vertical press for hand protection adjacent to the fingers being moved.

Eight slings (4 for flexing and 4 for extension), equipped with bars with weights for programmable resistance for each metacarpal-phalange articulation required.

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Email: info@ortotec莎.com / Web: www.ortotec莎.com

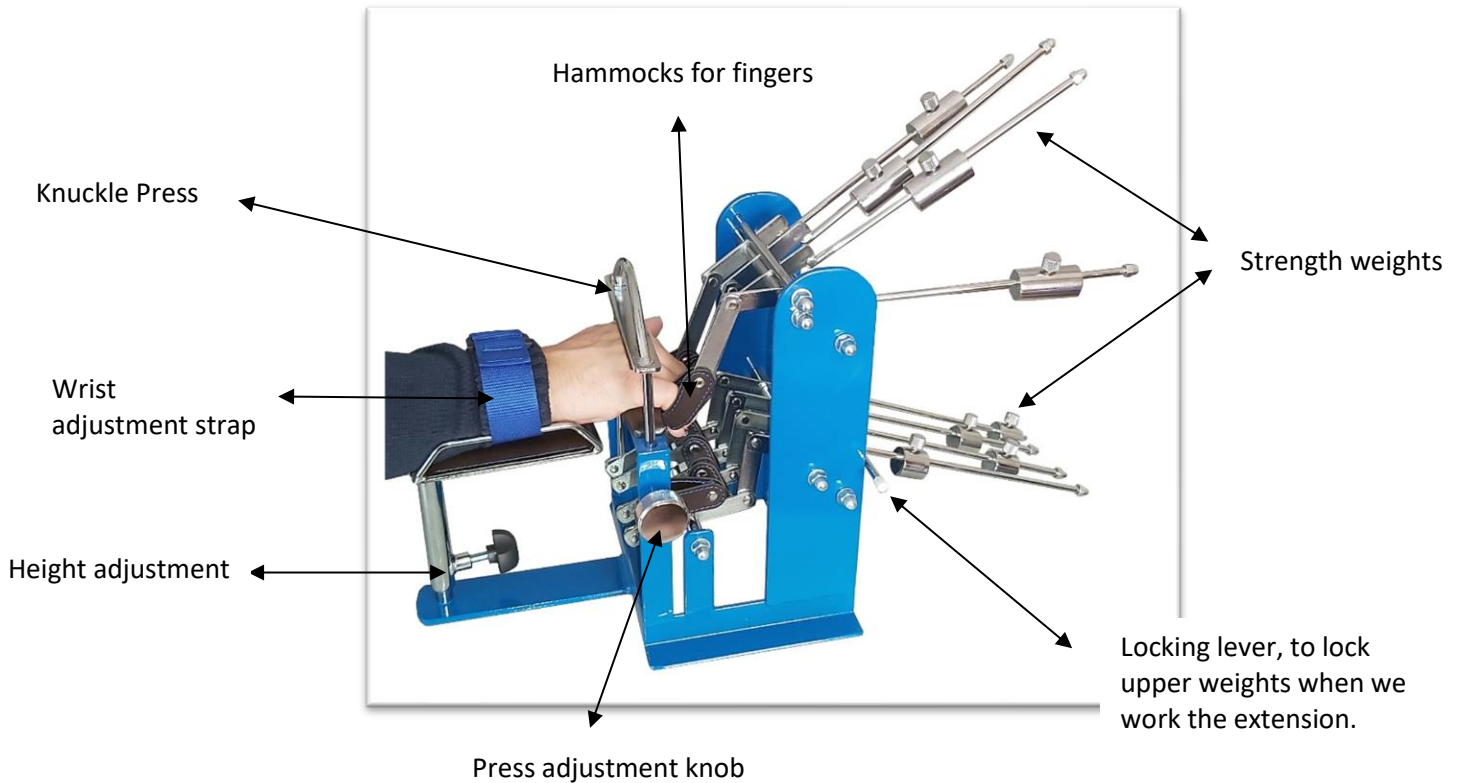
1. TECHNICAL SPECIFICATIONS:

Metallic component finished in chrome and polyester.

Hammocks for fingers and padded leather areas.

Measurements: 61x24x33 (Length x Width x Height) - cms.

2. PLACEMENT:

1. Place the arm horizontally on the armrest, leaving the knuckles positioned on the adjustment vise.
2. Tighten the wrist strap and tighten the knob on the adjusting vise to immobilize the hand.
3. Put your fingers in each corresponding hammock (upper when we want to do the flexion, lower when we want to do the extension).
4. Adjust the weights to the desired force.

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
Tlf: 0034 918 719 132 / Fax: 0034 918 719 323

Email: info@ortoteca.com / Web: www.ortoteca.com

- To work the flexion of the fingers (Figure A.) you must place the hammocks in the lower part of the fingers.

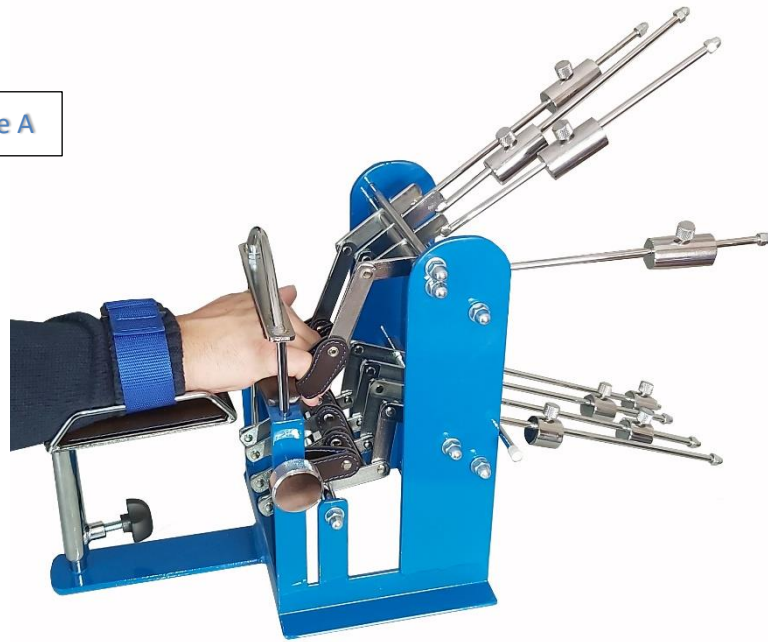Figure A

- To work the extension of the fingers (Figure B.) you must place the hammocks on the upper part of the fingers.

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Email: info@ortotecsa.com / Web: www.ortotecsa.com

3. INDICATIONS:

The purpose of the mechanized component is the rehabilitation of any injured finger, strengthening it and improving mobility, through flexion and extension exercises with resistance opposition.

- Flexion: Bend and straighten the injured finger (s), flexion constitutes most of the movement of the finger joints (See fig. A)
- The Extension: Raise the injured finger (s), the extension corrects the rigidity of the contracture of the wrist extensor muscle in patients with dysfunction of the finger and wrist (See fig B)
- To make a correct flexion or extension of the finger joints, the wrist must be placed and kept in a neutral position.

Some injuries that can be treated with this equipment:

- Ligament injuries (skier's thumb)
- Tendinitis (irritation of the tendons)
- Injuries to the tendons (hammer toe)
- Joint injuries (sprains)
- Strains (muscle pulls)
- Dislocations

4. WARRANTY:

ORTOTECOSA, S.L. will provide a warranty for two years against any manufacturing defect from the date of purchase. Our guarantee includes the repair, replacement or exchange of the product and / or components, including labor carried out in our facilities. To take effect this warranty, the purchase invoice is required.

This warranty will not be valid under the following conditions:

- When the use, care and operation of the product have not been carried out in accordance with the instructions indicated.
- When the product has been used beyond its capacity, mistreated, beaten, exposed to humidity, attacked by any liquid or corrosive substance, as well as any other fault attributable to the consumer.
- When the product has been disassembled, modified or repaired by unauthorized persons.

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