

PRONOSUPINADOR

SOBRE MADERA - F.42/46 **A PARED - F.42/47**

Caja mecanismo graduable de altura provista de empuñadura y mando para la oposición de resistencia.

Instalado sobre tablero de madera barnizada. (F.42/46)

Se instala directamente en pared. (F.42/47)

De construcción metálica rematado en cromo y poliéster.

PRONATOR-SUPINATOR

ON WOODEN BOARD – F.42/46 **WALL PRONOSUPINER – F.42/47**

Box mechanism with adjustable height.

Equipped with hand grip and control for resistance exercises.

Installed on a varnished wooden board model F.42/46.

Metal frame with chrome and epoxy finish.



RECOMENDACIONES DE USO:

Consiste en que el paciente agarre el asidero con la mano y la mueva continuamente realizando así la pronosupinación de las muñecas.

Con esta actividad se pretende mejorar la potencia y la resistencia muscular. Si objetivo es rehabilitar, mantener o aumentar la flexibilidad de la muñeca y del antebrazo y su movilidad.

RECOMMENDED USE:

Consists that the patient holds the grip with his hand and move continuously performing pronosupination of the wrist.

This activity is to improve power and muscular endurance. Its objective is to restore, maintain or increase the flexibility of the wrist and forearm and its mobility.

RECOMENDACIONES INSTALACIÓN / RECOMMENDED INSTALLATION:

