PARALELAS GRADUABLES CON PLANO SEPARADOR ADULTOS - F.50 / INFANTIL - F.54N

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F.50 - Longitud 350 cm.

F.54N - Longitud 250 cm.

PARALLEL BARS WITH FOOT PLACEMENT LADDER ADULTS – F.50 / CHILDREN – F.54N

A high degree of stability is ensured with the metal frame of the apparatus, the material used and a weighted base.

The height of the base off the floor has been reduced to a minimum to allow for easy access.

Height can be adjusted using pins that remain hidden on the upper part of the columns.

Adjustable width (can be folded up).

Equipped with abduction board, for guidance and to prevent feet crossing, easily-collapsible.

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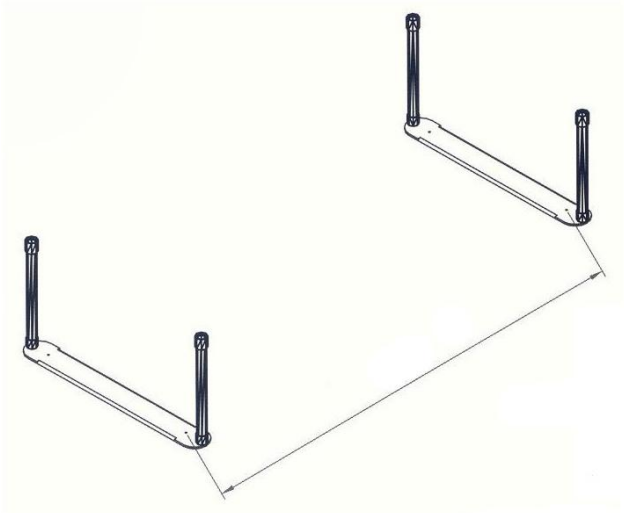
Must be installed in parallel to the walls of the gym but at enough distance so that the physiotherapist can move while the patient is walking. It will be place a mirror on the wall in one or both ends so that the patient might see himself and coordinated wandering. Passed this stage of wandering we can continue with ramp and stairs. Follow carefully all instructions given by your doctor, physic or therapist for the use of the product.

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

1 The parallels are supplied with hollow tubes in the weighted basis. Place the weighted basis at the distance indicated in the measure section of each parallel.

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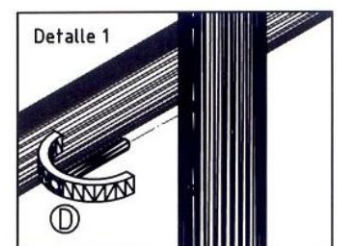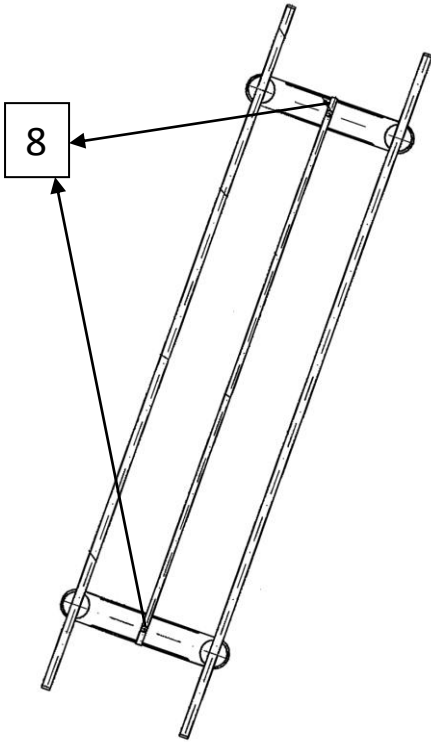2 Place the handrails inside the hollow tubes and select the desired height inserting the plugs in the appropriate holes (see detail)

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
3. To place the abduction board anchor to the weighted base with knobs (8).

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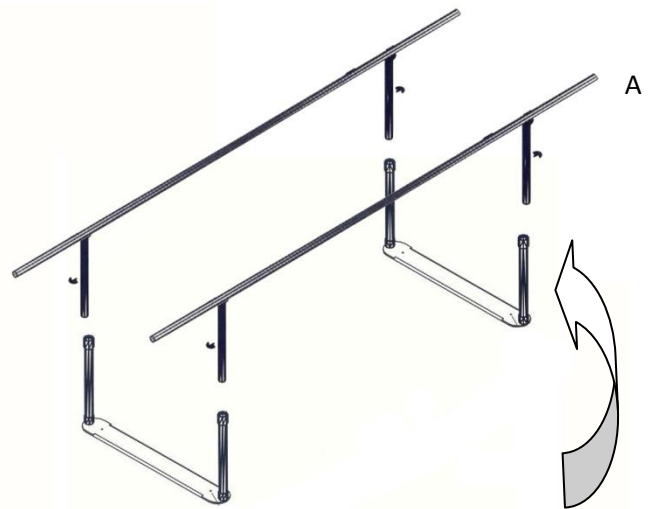NOTE. To fold the parallels remove the abduction board, then do it in the direction of the arrows as shown, bringing the handrail A into the handrail B.

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