PARALELAS GRADUABLES EN ANCHO

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Longitud 350 cm.

PARALLEL BARS (ADJUSTABLE WIDTH)

Same characteristics as model F.51, with adjustable width: from 60 cms. to 100 cms.

A high degree of stability is ensured with the metal frame of the apparatus, the material used and a weighted base.

The height of the base off the floor has been reduced to a minimum to allow for easy access.

Height can be adjusted using pins that remain hidden on the upper part of the columns. (Height 68-98 cms.)

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

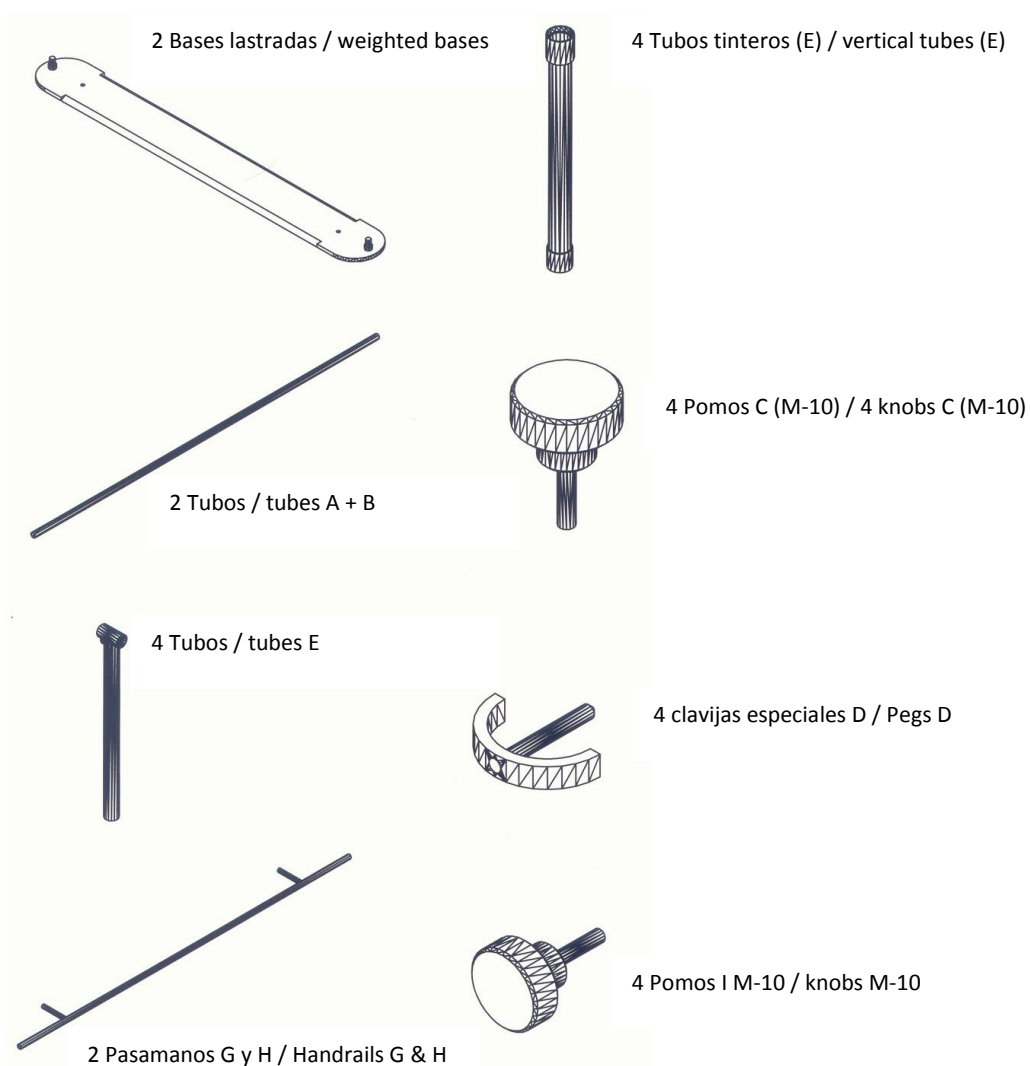
Must be installed in parallel to the walls of the gym but at enough distance so that the physiotherapist can move while the patient is walking. It will be place a mirror on the wall in one or both ends so that the patient might see himself and coordinated wandering. Passed this stage of wandering we can continue with ramp and stairs.

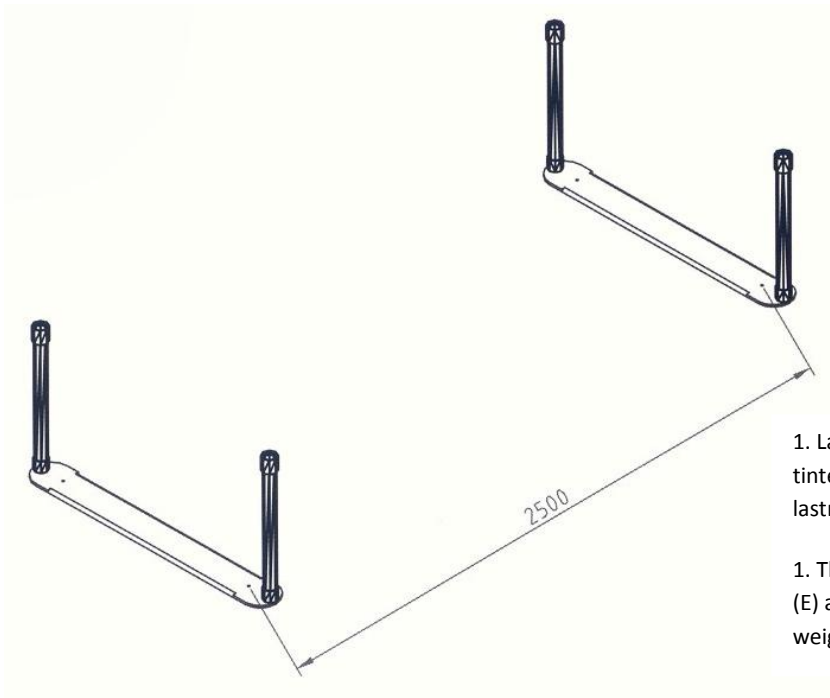Follow carefully all instructions given by your doctor, physio or therapist for the use of the product.

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
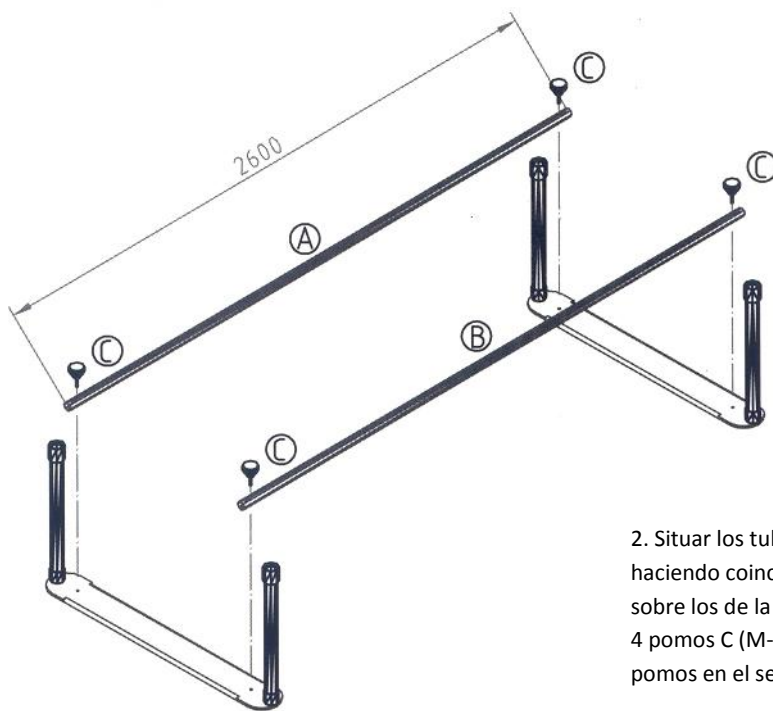
1. The parallels are supplied with the vertical tubes (E) assembled in the weighted bases. Place the weighted basis at a distance of 2500mm.

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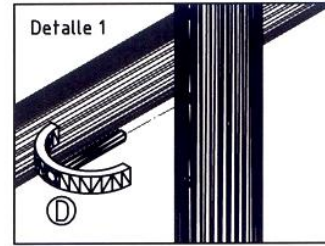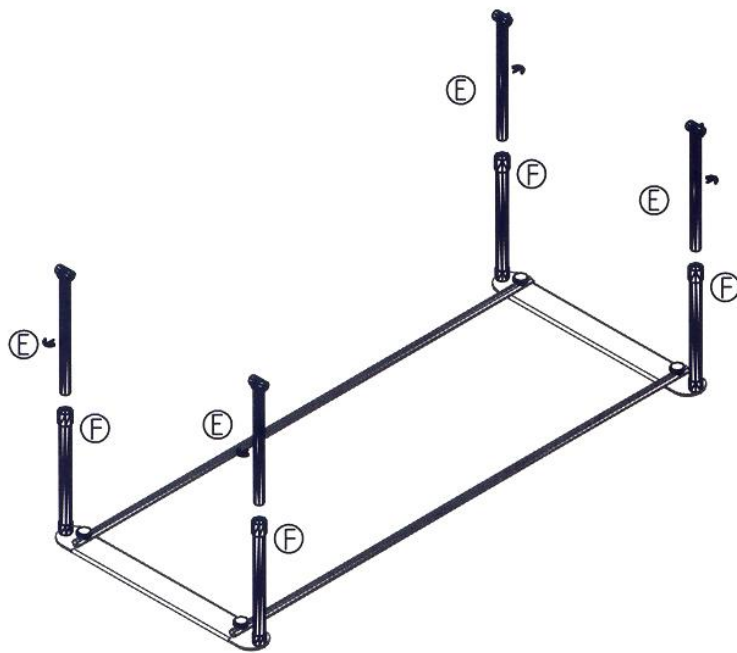
2. Place the tubes A and B (2600mm long) aligning the holes on the ends with the holes in the weighted base (E). Attach the tubes to the base by the 4 knobs C (M-10), turning the knobs clockwise.

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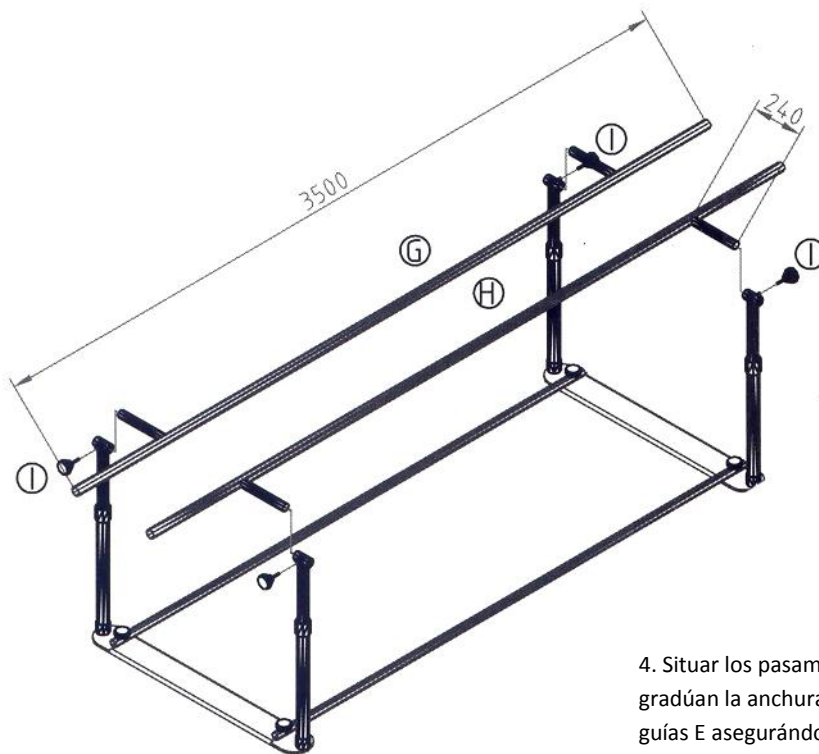3. Place the special chrome pegs D in the tubes which adjust the height (detail 1) by placing the rod of the peg in the desired hole; there are 7 holes with a separation of 50mm. Place the tubes E (585mm) in each guide tube F.

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4. Locate the handrails G and H (3500mm), which graduated the width of the parallel in the guide tubes, which graduated the height E, securing them through the 4 knobs supplied M-10.

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