## PARALELAS STANDARD - F.51-S

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Longitud 300 cm.

## STANDARD PARALLEL BARS – F.51-S

Adjustable height and width (until totally folded).

The height of the base off the floor has been reduced to a minimum to allow for easy access to the walkway.

Metal frame with chrome and epoxy finish.

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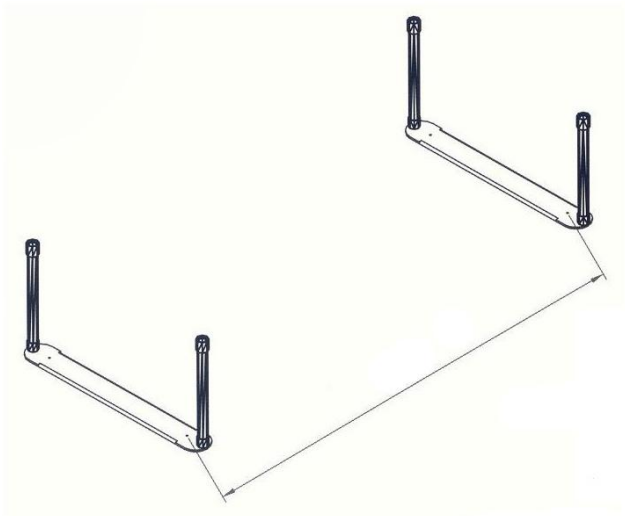
*Must be installed in parallel to the walls of the gym but at enough distance so that the physiotherapist can move while the patient is walking. It will be place a mirror on the wall in one or both ends so that the patient might see himself and coordinated wandering. Passed this stage of wandering we can continue with ramp and stairs. Follow carefully all instructions given by your doctor, physic or therapist for the use of the product.*

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*1 The parallels are supplied with hollow tubes in the weighted basis. Place the weighted basis at the distance indicated in the measure section of each parallel.*

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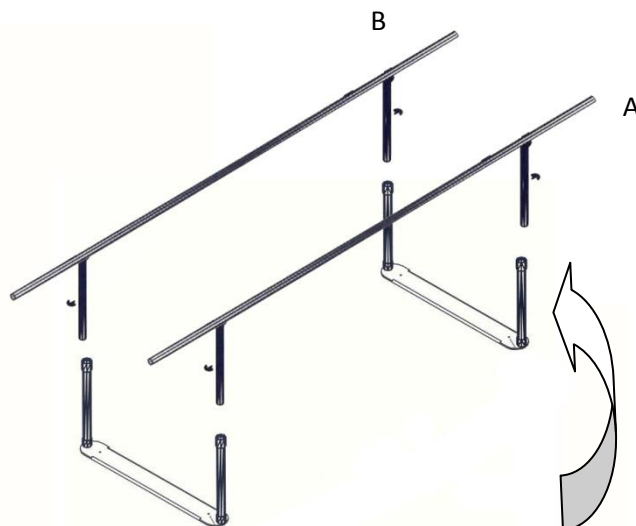
*2 Place the handrails inside the hollow tubes and select the desired height inserting the plugs in the appropriate holes (see detail)*

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*NOTE. To fold the parallels remove the abduction board, then do it in the direction of the arrows as shown, bringing the handrail A into the handrail B.*

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