**F.52 & 56N PARALELA GRADUABLE CON MONORRAÍL  
(PARALLEL BARS WITH FOOT PLACEMENT LADDER)**

Última revision 13/04/2013

# **PARALELAS CON MONORRAIL - F.52**

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# **INFANTIL- F.56N**

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# **PARALLEL BARS WITH MONORAI – F.52**

## **PARALLEL BARS WITH MONORAIL FOR CHILDREN – F.56N**

A high degree of stability is ensured with the metal frame of the apparatus, the material used and a weighted base.

The height of the base off the floor has been reduced to a minimum to allow for easy access.

Height can be adjusted using pins that remain hidden on the upper part of the columns.

Adjustable width (can be folded up).

Equipped with abduction board, for guidance and to prevent feet crossing, easily-collapsible.

Chrome and epoxy finish.

F.52 - Length: 350 cms. F.56N - Length: 250 cms.

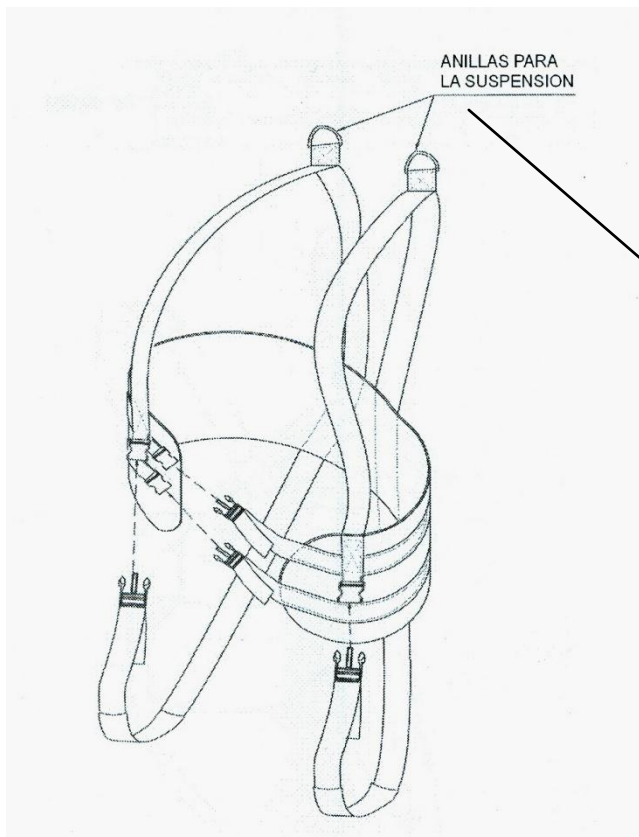
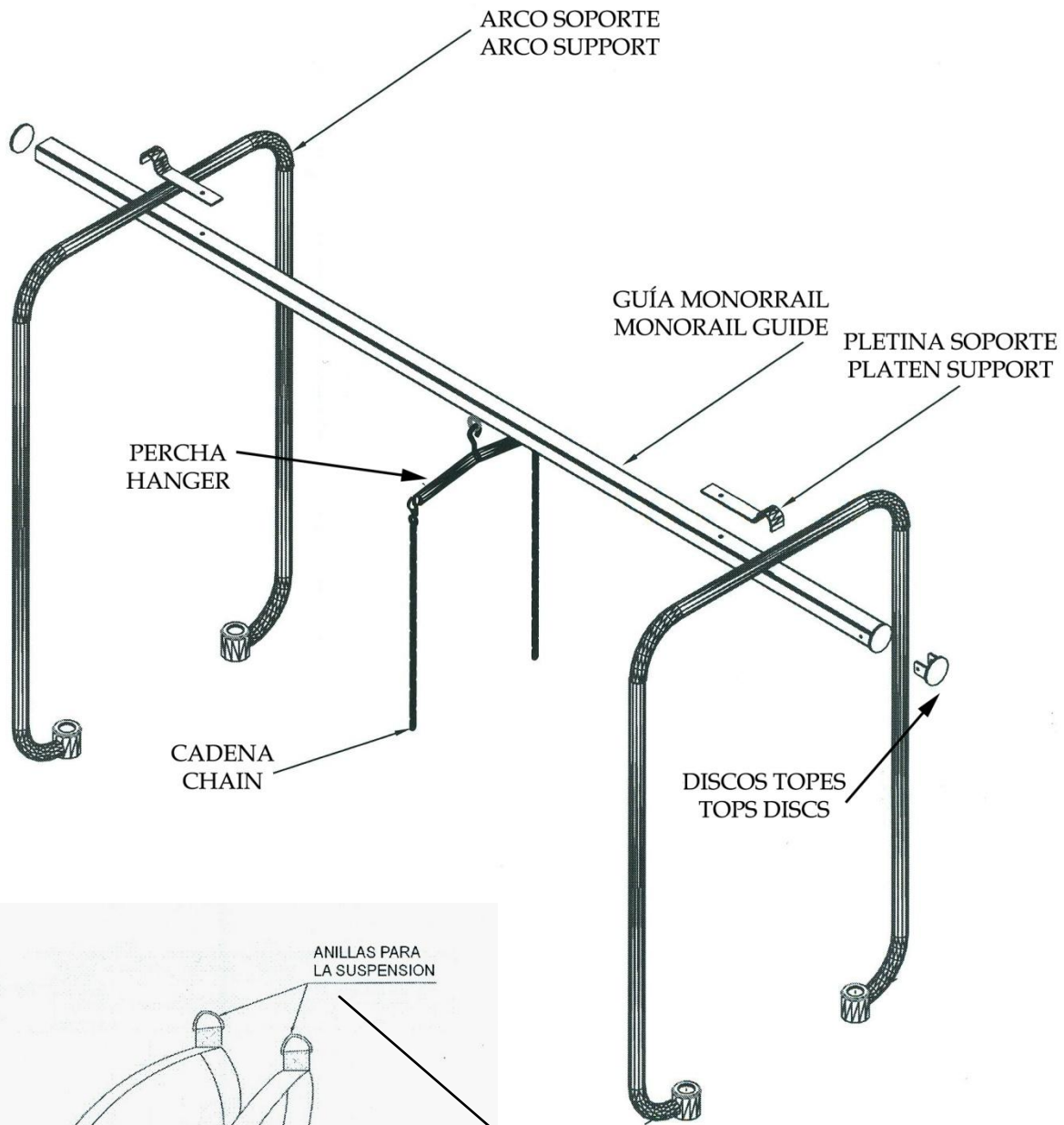
Removable suspension system and abduction board.

Ball bearing monorail traveller.

Provided with suspension harness for walking.

### **RECOMMENDATIONS FOR USE**

Must be installed in parallel to the walls of the gym but at enough distance so that the physiotherapist can move while the patient is walking. It will be placed a mirror on the wall in one or both ends so that the patient might see himself and coordinated wandering. Bars must be set solid to the ground so not to move or oscillate. Passed this stage of wandering we can continue with ramp and stairs. Follow carefully all instructions given by your doctor, physiotherapist or therapist for the use of the product.

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## (ASSEMBLY INSTRUCTIONS)

- 1) Mount the arches on the inkwells (it will be subject immediately).
- 2) Put the handrail of the parallel bars on the inkwells to the desired height, setting the pin into the hole corresponding to that height.
- 3) Hold the monorail to the arcs by two screws on each of the two platen support on each side of the monorail.
- 4) Insert the hanger by the monorail by one side of the tops discs and hang the harness.
- 5) The foot placement bar will go on weighted base with two screwed accompanying.