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F.53 - Longitud 350 cm.

F.58N - Longitud 250 cm.

PARALLEL BARS WITH FOOT PLACEMENT LADDER ADULTS – F.53 / CHILDREN – F.58N

With the metallic frame of the apparatus and the section of material used, being the motherboards weighted base, a high degree of stability is achieved.

The height of the base off the floor has been reduced to the maximum, facilitating the way to the walk area.

Adjustable in height using pins, that remains hidden in the upper part of the columns.

Adjustable width until being folded up.

All the components forming the foot placement ladder are dismantled to facilitate the folding of the parallel (F.53)

For his disposal, it is allowed to disassemble each of its components (track and abduction board) which facilitates the folding of the same (F.58N)

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Must be installed in parallel to the walls of the gym but at enough distance so that the physiotherapist can move while the patient is walking. It will be place a mirror on the wall in one or both ends so that the patient might see himself and coordinated wandering. Bars must be set solid to the ground so not to move or oscillate. Passed this stage of wandering we can continue with ramp and stairs.

Follow carefully all instructions given by your doctor, physic or therapist for the use of the product.

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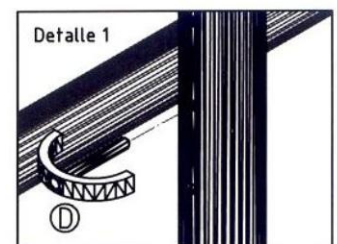
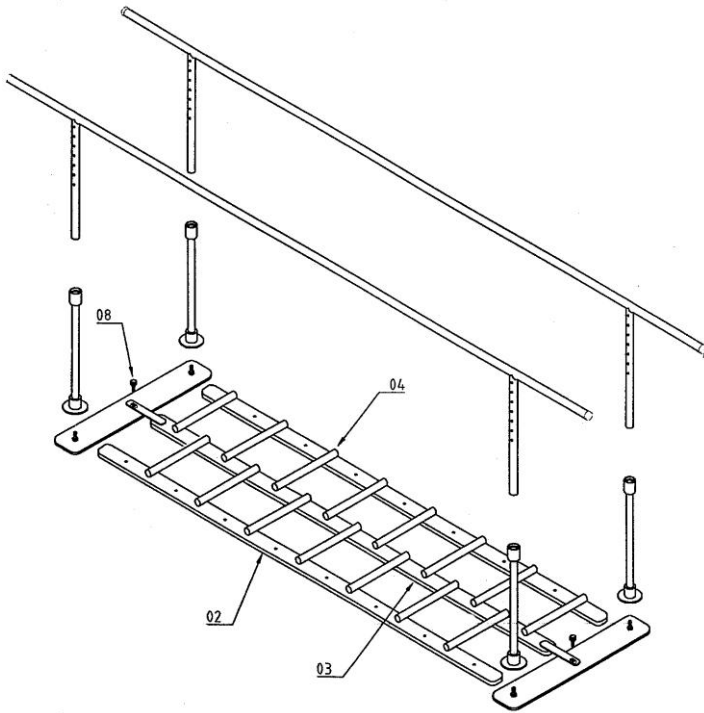
1 The parallels are supplied with hollow tubes in the weighted basis. Place the weighted basis at the distance indicated in the measure section of each parallel.

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2 Place the handrails inside the hollow tubes and select the desired height inserting the plugs in the appropriate holes (see detail)

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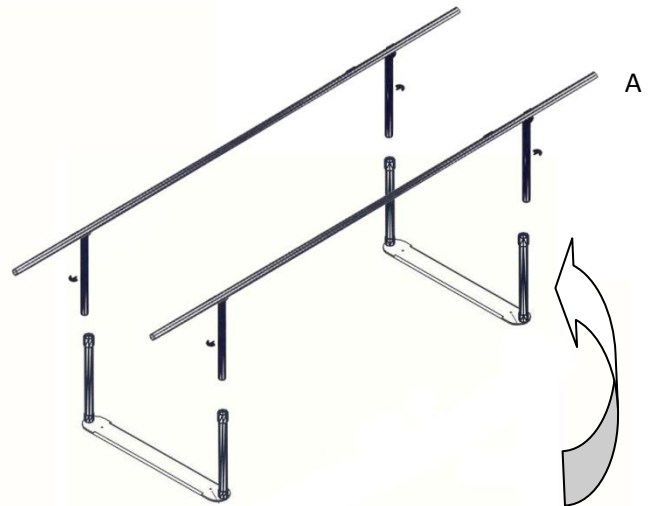
3. To place the obstacle track, we must first put the central base and anchor to the weighted base with knobs (8). Then place in parallel on either side the two remaining bases (2) and be placing every obstacle (4) into the desired holes.

In the child model (F.58N) obstacle track is already assembled to put on the central abduction board. This track can be placed at the desired height.

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NOTE. To fold the parallel remove the gridded tracks and the abduction board, then fold in the direction of the arrows as showing, bringing the handrail A into the handrail B.

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