BAÑOS DE CONTRASTE

EXTREMIDADES SUPERIORES - G.18953 CON TERMÓMETRO - G.18953/A EXTREMIDADES INFERIORES - G.18954 CON TERMÓMETRO - G.18954/A

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G.18954 & G.18954-A Medidas: L=920, F=610, A=560 mm.

CONTRAST BATH

UPPER EXTREMITIES – G.18953 WITH THERMOMETER – G.18953/A LOWER EXTREMITIES – G.18954 WITH THERMOMETER – G.18953/A

Made from stainless steel AISI 304 (nonferric).

Two sinks, with a mix tap each.

Drain and overflow into the same unit.

Dimensions: L=920, F=610 and A=900 mm.

Each sink has a thermometer acting as a constant temperature control. MODEL:

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
RECOMMENDATIONS FOR USE

Involves the application of alternately baths with hot and cold water. This temperature difference will produce cutaneous vasodilatation (hot water - opening of blood vessels) followed by cutaneous vasoconstriction (cold water – vessel closure) that would stimulate the local blood circulation. This will cause the inflow increased blood supply to the area, improving nutrition and damaged tissue oxygenation as well as better return circulation and cleanse of the waste produced by inflammation.


We will have two containers, one with hot water between 38 and 44 degrees, and the other one with cold water between 10 and 20 degrees.

The inflamed joint will submerge in hot water for about 3 to 6 minutes then switch to cold water for 1 to 3 minutes, starting and ending with the cold water as we seek to reduce inflammation making a total of 3 to 5 cycles of contrast (cold-hot). During immersion in hot water we can use to move the affected joint seeking to gain motion helping with the relaxation that causes the high temperature (but without much force) and during cold immersion, keep the articulation immobile.

Follow carefully all instructions given by your doctor, physic or therapist for the use of the product.

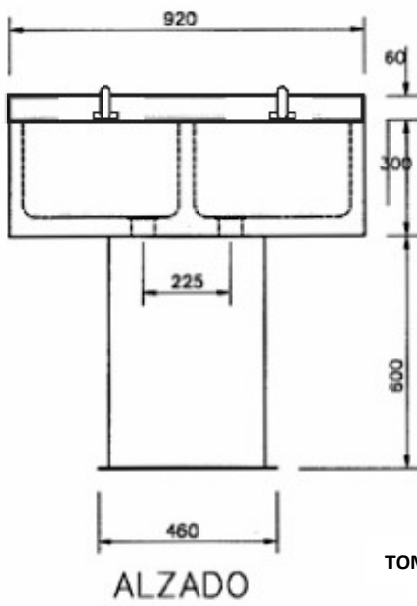


TOMA DE AGUA
1/4"

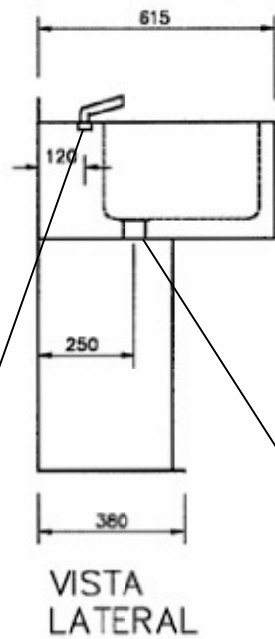
DESAGÜE
Ø 40mm

G.18953
&
G.18953-A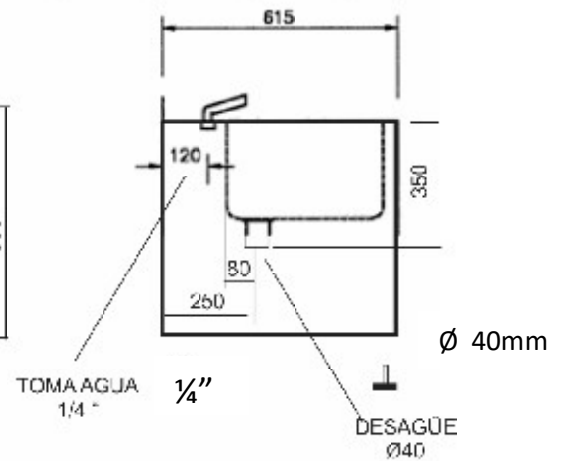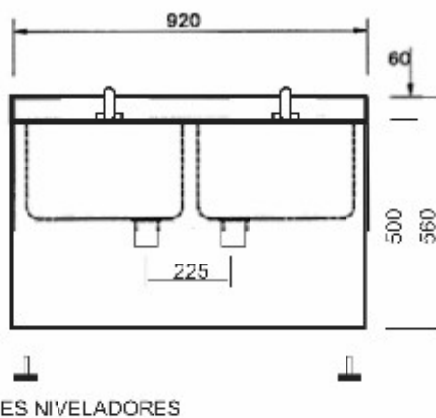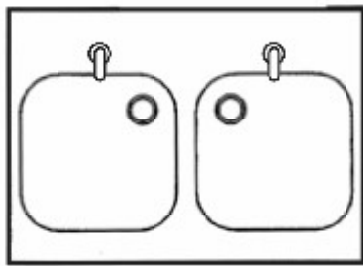

G.18954
&
G.18954-A

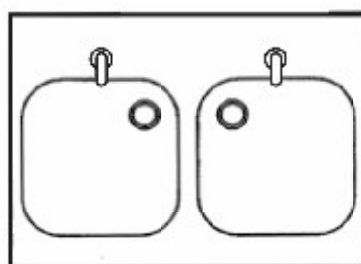PLANTA