CARRO DE LAVADO ELECTRICO - O.9200-E

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TROLLEY FOR WASHING ELECTRIC – O.9200-E

Adjustable height to place the trolley at the height of the bed, greatly simplifying the carers job.

Collapsible side bars, to assure patient movements are reduced to a minimum.

Collapsible head rest.

Reclining upper section, using gas cylinder to allow proper drainage and cleaning of the mat.

Air tight, plastic-coated mat, equipped with drainage.

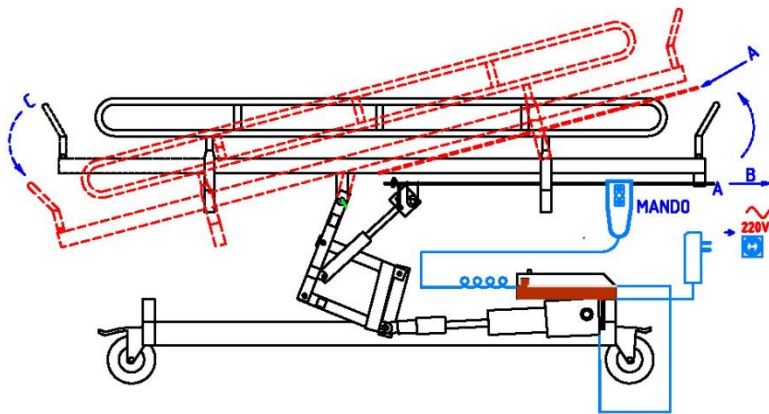
Four casters and a brake.

Metal frame, with chrome and epoxy finish.

Stainless steel frame.

OPTIONAL: MATTRESS OF TROLLEY FOR WASHING - O.9200/1

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TILT THE PLANE

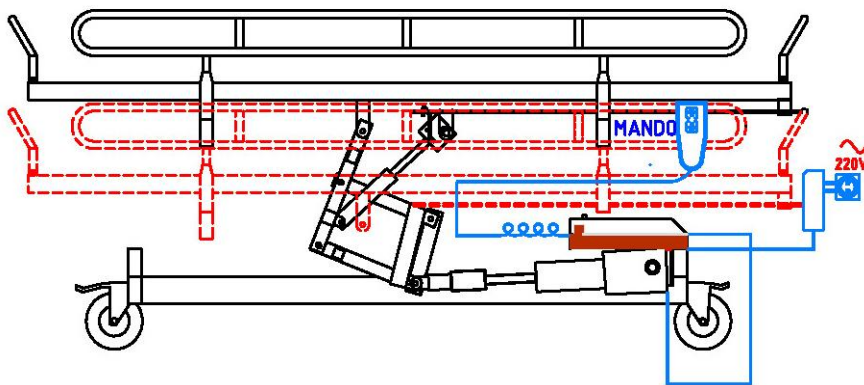
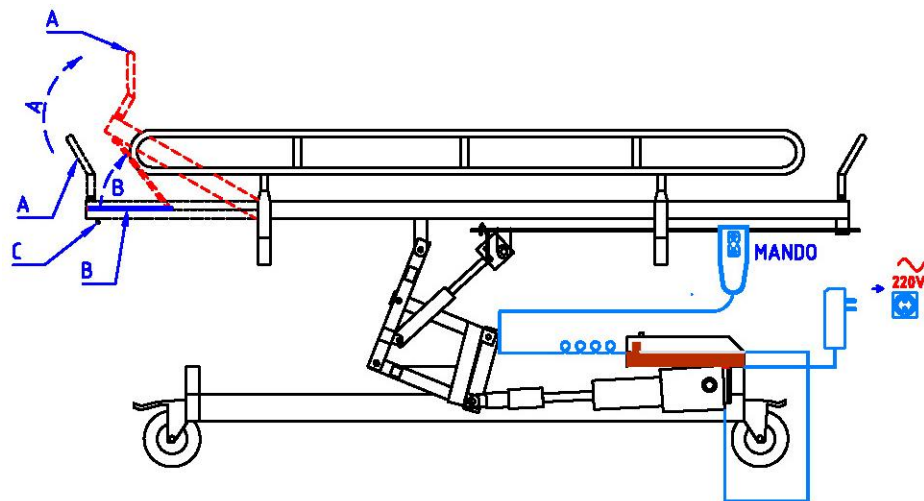
- 1º activate the lock position (A) by sliding in the direction of the arrow (B)
- 2º holding down the lock A, proceed to tilt the plane according to arrow (C) until desired position and
- 3º release the lock position (A) of the plane.

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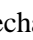
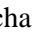
RAISE THE HEADBOARD

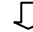
- 1º fold the headboard (A) in the direction of the arrow (A)
- 2º swing the rod (B) in the direction of the arrow (B)
- 3º fit the rod (B) in the flange (C).

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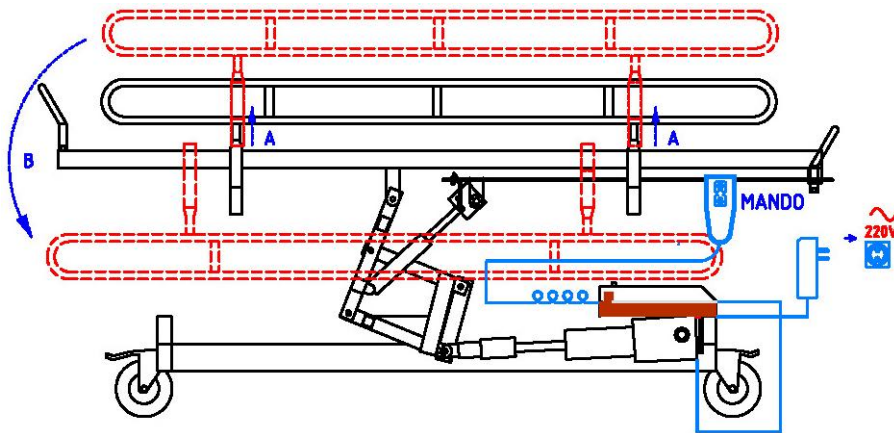
HEIGHT REGULATION

- 1º to raise the cart press the remote control on the arrow  until the desired height.
- 2º to lower the cart press the remote control on the arrow  until the desired height.

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FOLD RAILINGS

1º lift the rail according to the arrows (A) to release the pins fixed to the frame
 2º fold the railing in the direction of the arrow (B) perpendicular to the ground.
Important: Do not fold down the rails with the mat clutching them.

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The first battery charge

When the red light above the buffer switch on, put the charger to the main 220V A, until the green light indicate the end of the charge (16h approximately).

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VERY IMPORTANT!!!
RECHARGE THE BATTERIES EVERY NIGHT

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