STANDING DE POSICIONAMIENTO T.O.175

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STANDING FRAME – T.O.175

Height-adjustable.

Tray adjustable in height and proximity.

Set of hip stabilisers and pelvic buffer, adjustable leg stop.

Non-slip feet rest with adjustable guard and strap.

Metal frame, with chrome and epoxy finish.

All components cushioned with fireproof upholstery.


Sizes: Width 54 cms. Depth 37cms (without pelvic support) to 54 cms.

OPTIONAL: ADJUSTABLE PELVIC SUPPORT – T.O.175/A

USER INSTRUCTIONS


Manufacturer: Ortotecsa,s.l. C/Peña de Francia,8 – 28500 Arganda del Rey (Madrid), España.
Tlf.: +34918719132 / Fax: +34918719323 – email: info@ortotecsa.com, www.ortotecsa.com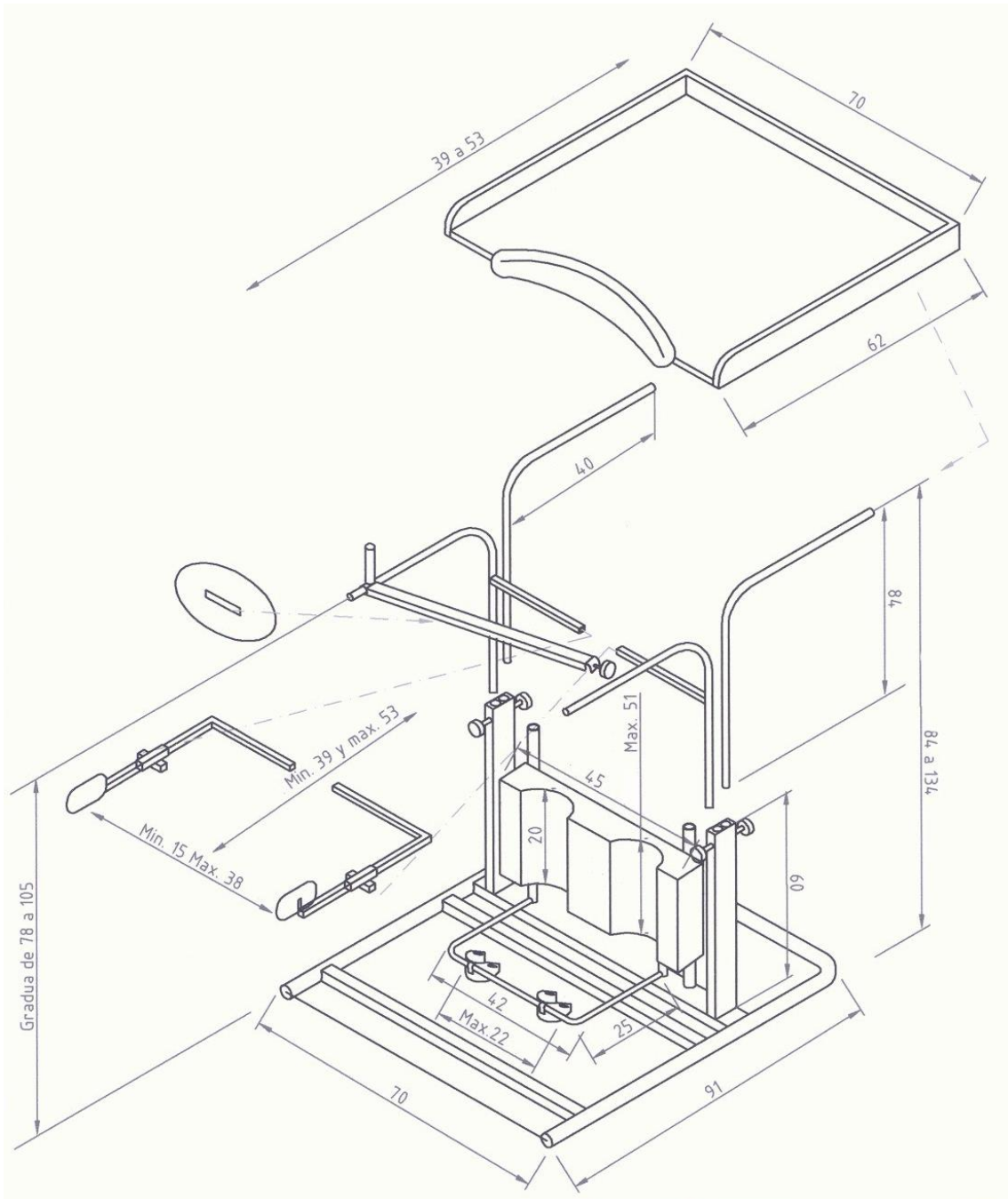


INSTRUCTIONS:

First, it's necessary to adjust all the accessories of the standing to the correct size for the user. (See adjustment accessories). After making adjustments, you can use the standing.

Access: Fold down the backrest in the direction shown by the arrow F / A, place the feet in the cups and close the backrest by fist M as indicated by the arrow.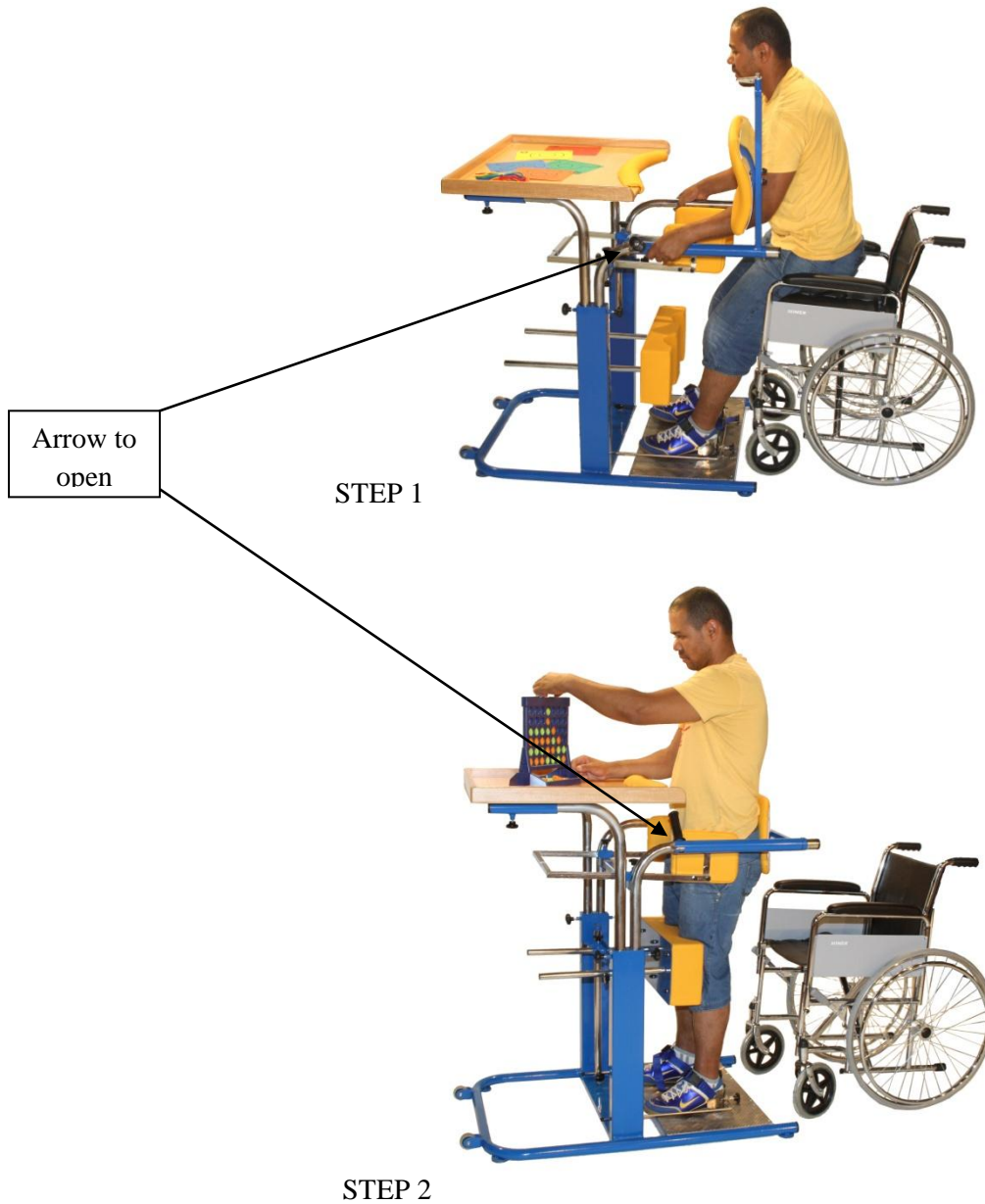

ADJUSTMENT OF ACCESORIES:

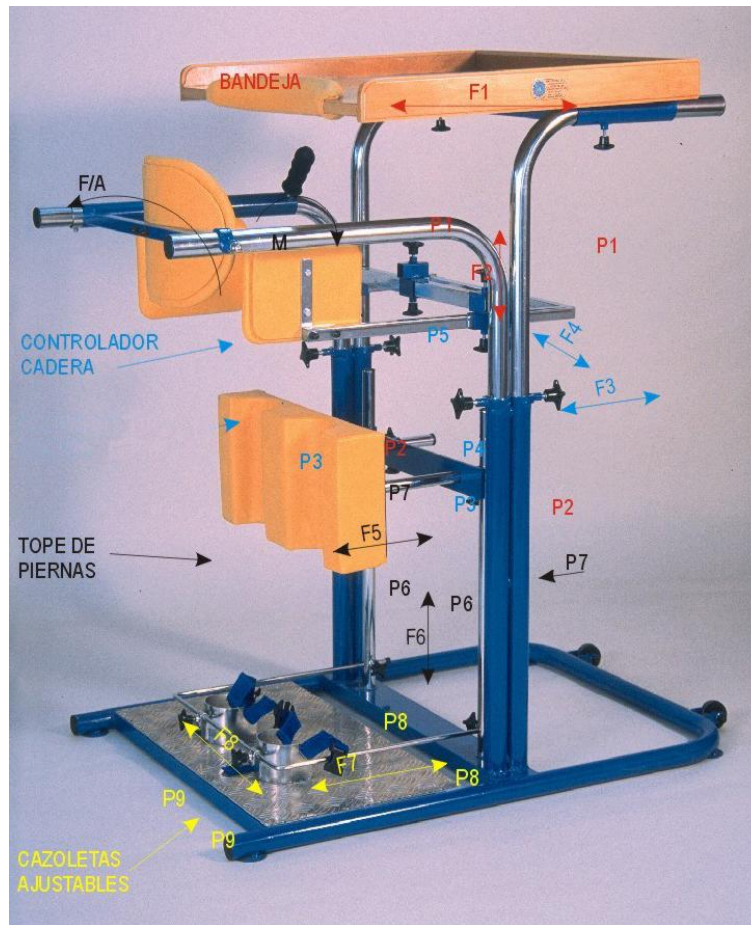- Adjustment Tray (BANDEJA):

Adjustable height and approach.

Height: minimum 85 cms and maximum 140 cms.

Approach: minimum 39 cm and a maximum of 53 cms.

Height: Loosen pommels P2, slide in the direction of arrow F2 to the desired position. Fix it tightening the P2 pommels again.

Depth: Loosen pommels P1, slide in the direction of arrow F1 to the desired position. Fix it tightening the P1 pommels again.

- Adjustment pelvic support (CONTROLADOR DE CADERA):

Adjustable height, width and approach.

Height: minimum 70 cms and maximum 97 cms.

Width: minimum 15 cm and a maximum of 38,5 cms.

Approach to the tray: minimum 39 cm and a maximum of 53 cms.

Height: Loosen pommels P3, slide in the direction of arrow F2 to the desired position. Fix it tightening the P3 pommels again.

Depth: Loosen pommels P4, slide in the direction of arrow F3 to the desired position. Fix it tightening the P4 pommels again.

Width: Loosen pommels P5, slide in the direction of arrow F4 to the desired position. Fix it tightening the P5 pommels again.

- Adjustment top for legs (TOPE DE PIERNAS):

Height: minimum 25 cms and maximum 51 cms.

Height: Loosen pommels P7, slide in the direction of arrow F6 to the desired position. Fix it tightening the P7 pommels again.

Depth: Loosen pommels P6, slide in the direction of arrow F5 to the desired position. Fix it tightening the P6 pommels again.

- Adjustment cups for feet (CAZOLETAS AJUSTABLES):

Maximum width of separation between cups of 22 cms.

Approach maximum of 42 cms.

Depth: Loosen pommels P8, slide in the direction of arrow F7 to the desired position. Fix it tightening the P8 pommels again.

Width: Loosen pommels P9, slide in the direction of arrow F8 to the desired position. Fix it tightening the P9 pommels again.

- Adjustment backrest:

Height: minimum 78 cms and maximum 105 cms.