# ESTABILIZADOR-BIPEDESTADOR:

DE 2 A 8 AÑOS - T.P.1006

DE 6 A 12 AÑOS - T.P.1006/B

+ DE 12 AÑOS - T.P.1006/C

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## **STANDING FRAME:**

**FROM 2 TO 8 YEARS - T.P.1006**  
**FROM 6 TO 12 YEARS - T.P.1006/B**  
**+ OF 12 YEARS - T.P.1006/C**

All components adjustable. Padded leg positioning buffer.

Feet stabilising guard with straps, adjustable in width and proximity.

Adjustable straps.

Support base in varnished wood, equipped with 4 casters and brakes.

Metal frame, with chrome and epoxy finish. Fireproof upholstery.

### OPTIONAL ACCESORIE:

T.P.1006-1 - ADJUSTABLE TRAY

T.P.1006/B-1 - ADJUSTABLE TRAY

T.P.1006/C-1 - ADJUSTABLE TRAY

INSTRUCCIONES DE USO / USER MANUAL


Manufacturer: Ortotecsa,s.l. C/Peña de Francia,8 – 28500 Arganda del Rey (Madrid), España.

Tlf.: +34918719132 / Fax: +34918719323 – email: [info@ortotecsa.com](mailto:info@ortotecsa.com), [www.ortotecsa.com](http://www.ortotecsa.com)

### **ELEMENTS:**


1. Base
2. Wheels
3. Brake of the wheels
4. Fork holding cups
5. Cups
6. Straps
7. Locking knob approximation
8. Knob for height regulation
9. Height regulation rod
10. Knee support
11. Fixation height knob for knee support
12. Pelvic restrain strap
13. Abdominal restrain strap
14. Adjustable tray (optional)
15. Regulator tub tray height
16. Setting knob tray height
17. Tray support tube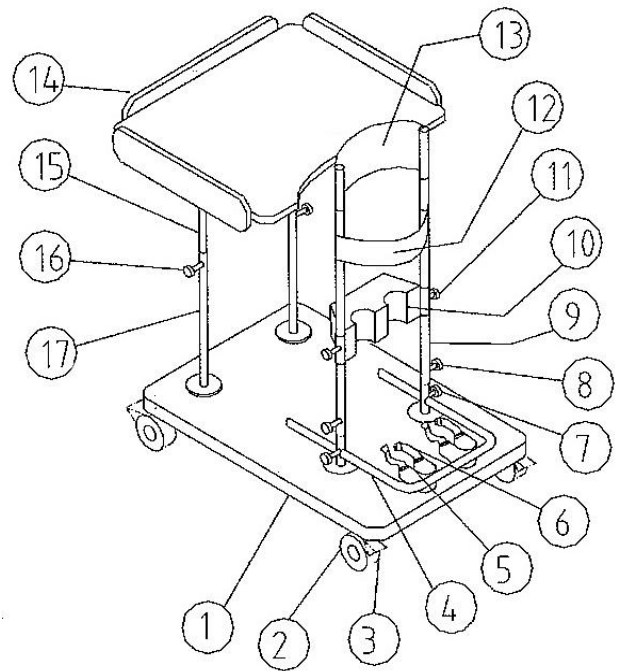

### **ASSEMBLY INSTRUCTIONS (TRAY):**

The T.P.1006 are supplied ready for use and require no assembly, except the adjustable tray.

To set the said tray (14) introduce the tubes (15) of the tray in the support tubes (uprights) (17). Then tighten the knobs (16) and after graduate to the desired height. Note that the red mark on the tube (15) is inside the support tube (17).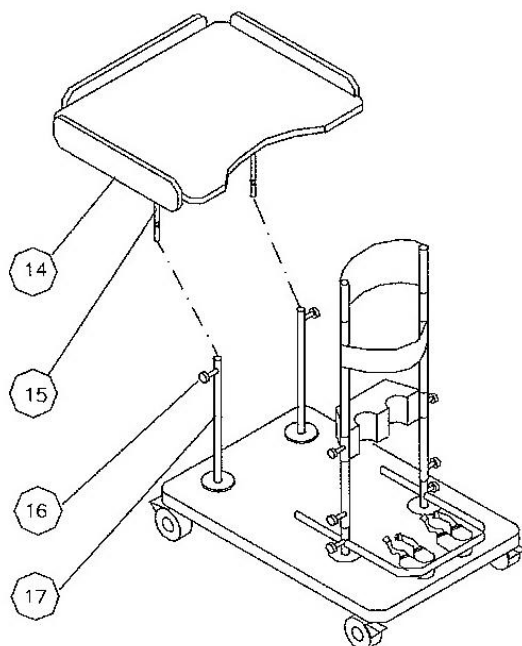

### **HOW TO USE:**

Before place the child in the stabilizer, operate the brakes (3) by pressing the lever under the wheels (2). First adjust the height of the main tub (9) fixing it by the knobs (8), adjust the straps (12 and 13), after the knee support (10) placing these in the support center. Next regulate the approximation of the feet guards for the feet (5) sliding the fork (4), tighten the knobs (7) and adjust the straps (6). Regulate the height of the optional tray (1) by sliding the tubes (15) on support tubes (17) by setting the knobs (16).

The final part of the regulations is indicated by a red mark on the tubes that slip inside. It is not recommended to exceed that mark for security. To move the stabilizer, once the child is properly secured, release the brakes (3) of each wheel pushing the levers upwards.